K-MOD™
KNEE-MODULAR SOLUTION

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The K-MOD system is a synthesis of innovations in total knee replacement that have proven their clinical efficacy. It's a complete modular system thanks to a suite of possibilities designed to satisfy the surgeon's approach to each patient.

The fixed insert models come in versions for conservation of the posterior crossed ligament (CR), UltraCongruent (UC), the development of the medial pivot concept: Dynamic Congruence (DC) and posterior stabilized (PS FXD). In the mobile insert version, the implants come as an UltraCongruent rotating platform (UC MOB) and posterior stabilized (PS MOB).

All the femoral and tibial designs are available in cemented, uncemented and anallergic versions; all the articular surfaces, femoral and tibial, are mirror polished.

The system comes complete with a patellar component and additional stems for the tibial component.

The instrument set is modular, compact and easy to use. The operating technique provides for the initial creation of the extension gap, thus enabling the surgeon to start with a tibial or distal femoral approach. Technologies are available for personalized 3D preoperative planning or for a completely extra-medullary approach. Such solutions meet specific patient needs and minimize complications like excessive wear, instability and the risk of patellar dislocation.

Thanks to full modularity with K-MOD REV revision implants, the system makes it possible to deal with a wide range of situations, being designed to provide less invasivity, bone sparing, proper alignment, balancing of the extension/flexion gaps and the restoration of knee biomechanics.

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On the patient's most recent X-rays, perform thorough pre-op planning on the AP and ML planes by using the X-Ray templates K-MOD: K-MOD X-Ray Template Fem AP/ML, Tib AP/ML, Fix/Mob, Stem (KMOD-XR-000÷008; KMODPS-XR-000÷003).

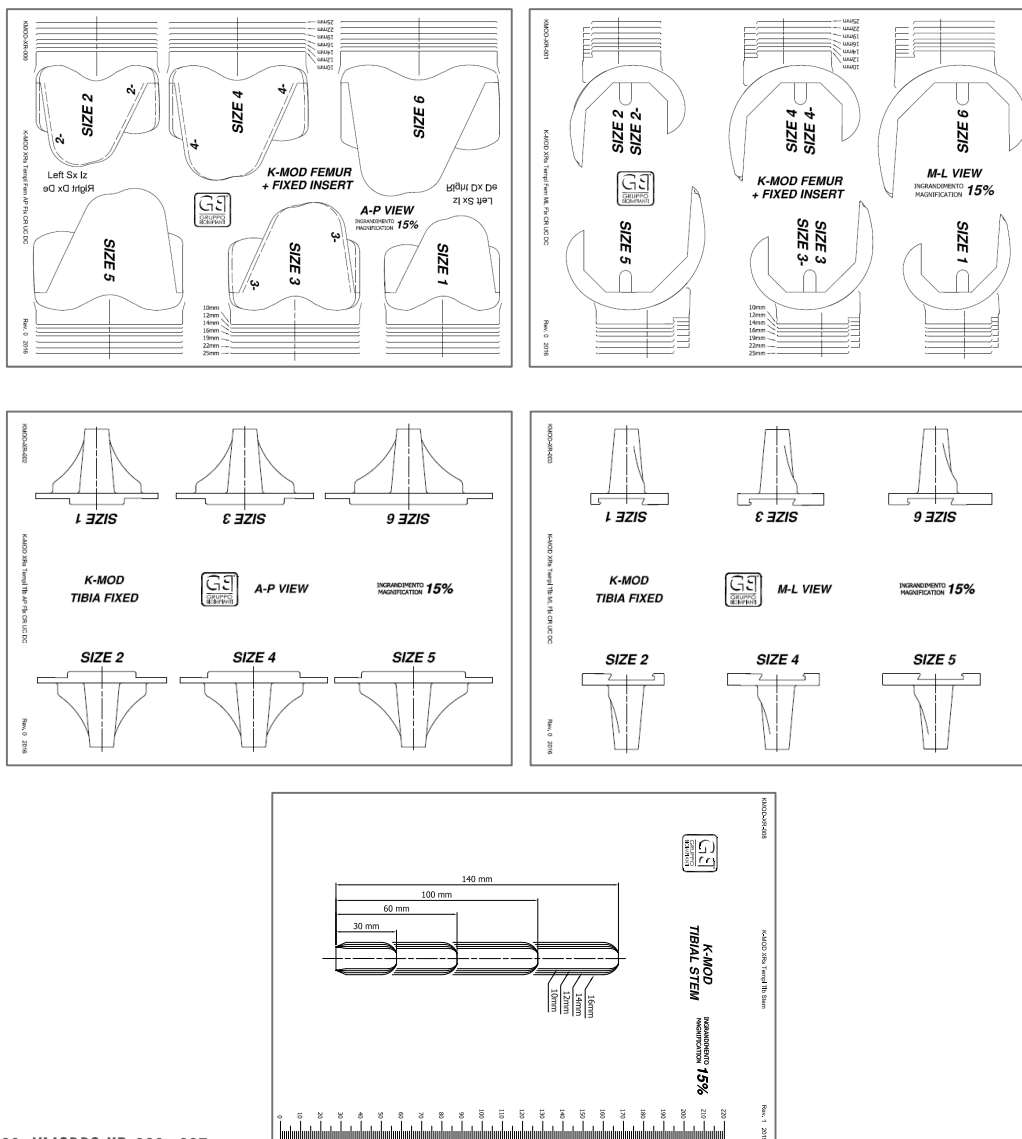
Evaluate (magnification: 15%): sizes of the femoral and tibial components, joint line, diameter and length of the optional tibial stem if needed.

Evaluation is indicative and must always be checked and modified during surgery if necessary.

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To simplify the nurse's intra-op tasks, the instrumentation develops from a minimum of only four trays with the following logical sequence:

Two basic trays:

- TIBIAL INSTRUMENTS (300110003);
- FEMORAL INSTRUMENTS (300110001);

Choice of the femur:

- FEMORAL TRIAL (300110002);
- PS FEMUR INSTRUMENTS (300111010).

Choice of the insert:

Fixed bearing:

- K-MOD TIBIA CR TRIALS (300110007);
- K-MOD TIBIA UC TRIALS (300110007);
- K-MOD TIBIA DC TRIALS (300110005);
- K-MOD PS FIXED TRIALS (300111003).

Mobile bearing:

- K-MOD TIBIA UC MOB TRIALS (300110004);
- K-MOD PS MOBILE TRIALS (300111002).

Three optional trays:

- PATELLA INSTRUMENT (300110016);
- STEM TRIAL (300110017);
- K-MOD SPACER BLOCK FLEX EXT-ROT (300112100).

SAW BLADES

Resection slots do accept saw blades up to 1,47mm height.

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PIN

The instrument set includes different types of pin. Unless specified otherwise, preferably use the Pin Dia 3,2 X 80mm (300110144).

Preferably insert the pin with the motor instead of the hammer, using the appropriate Jacobs Adaptor (300110377).

Always proceed by inserting the pin into the hole of the cutting guide by hand, even until contact with the bone, and then proceed to penetration into the bone, taking care to keep in line.

We suggest having around 6/8 pins available.

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Perform access to the knee joint based on the approach deemed appropriate by the surgeon.

The K-MOD TOUCH surgical technique preferably develops from the extension gap and allows starting with the proximal tibial or femoral distal resection, according to the operator surgeon's preference.

In the approach, consider the following features and possibilities:

- it is possible to perform proximal tibial resection in an extra or intra-medullary way;
- in case of Posterior Cruciate Ligament sacrifice or use of an additional tibial stem, do not exceed the posterior inclination of the tibial resection (0-3°);
- technologies are available for a femoral extra-medullary approach;
- the reference for the femoral measurement is: anterior;
- it is possible to perform the final verification of the most suitable insert even with the already implanted femoral and tibial components
- the system is completely compatible in all its versions: CR, UC, DC and PS.

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INSTRUMENT PRE-SETTING RECOMMENDATIONS. These indications reflect common surgical practice. They are intended not to be binding but to facilitate the pre-setting of certain instruments, especially for the instrumentation nurse.

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Make a longitudinal incision of the quadriceps to facilitate eversion of the knee cap and achieve adequate knee flexion.

The extra-medullary tibial resection guide consists of four parts: the Spikeless Tibial Resection Proximal Guide (300110115), the appropriate Tibial Cutting Block (RIGHT 300110128, LEFT 300110131), the Tibial Distal Guide (300110122) and the Ankle Clamp (300110133).

The guide must be assembled before being applied.

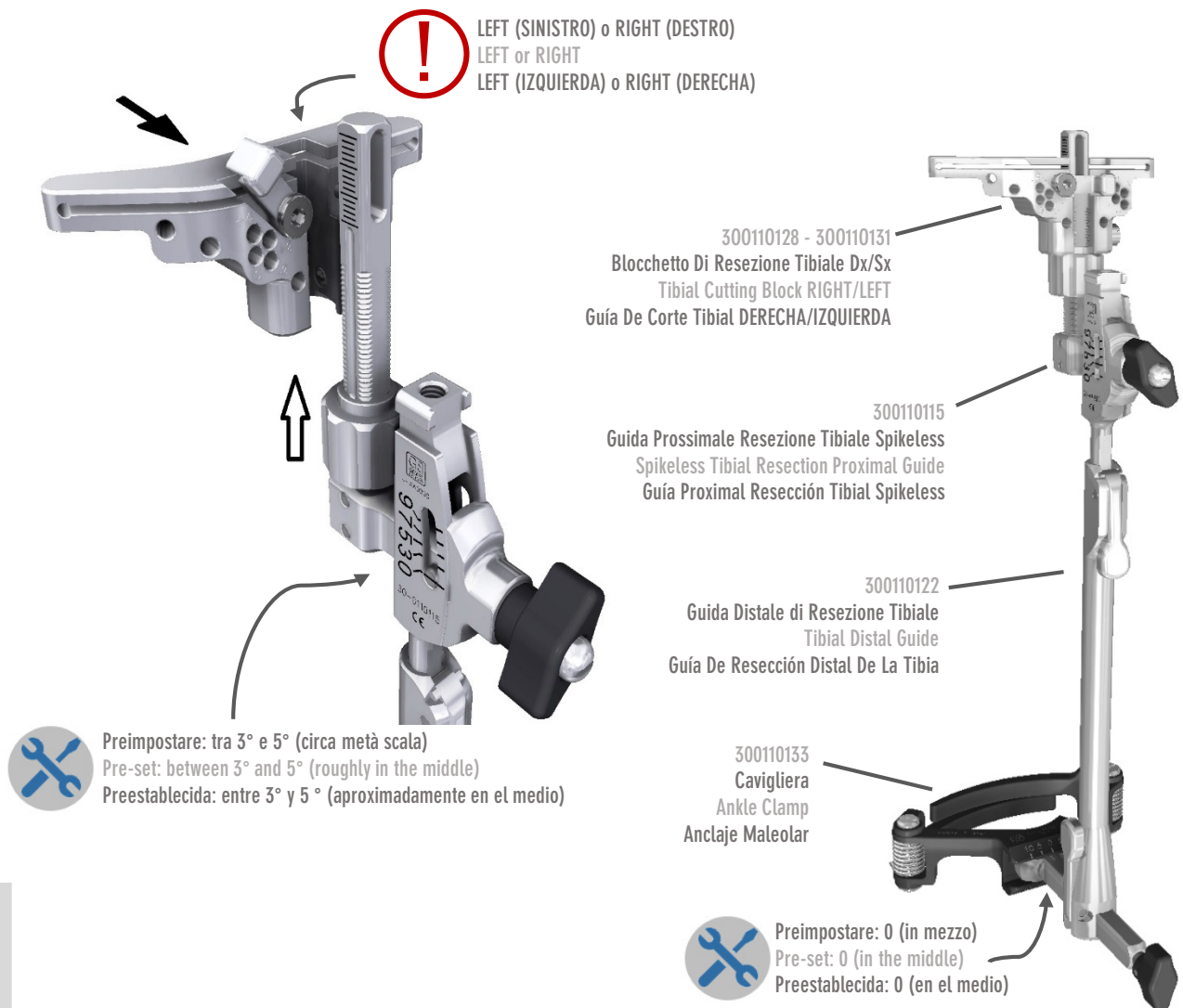
For the intra-medullary version, see the section entitled OPTION: INTRAMEDULLARY TIBIAL RESECTION.

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Make sure you have chosen the correct Tibial Cutting Block (LEFT, RIGHT). Preselect the desired tibial slope value (range: 0°-9°) on the Spikeless Tibial Resection Proximal Guide.

Enclose the malleoli with the Ankle Clamp and adjust the length of the guide so that the Tibial Cutting Block is at roughly the desired cut height. Align the tibial resection guide on the proximal tibia with the third medial of the tibial tuberosity and/or distally with the second metatarsophalangeal bone. In the sagittal plane, the guide must be parallel to the anterior tibial cortex.

Lock it in this position using a temporary Pin 3,2mm x 80mm (300110144) in the middle of the proximal slot. The pin will in any case make it possible to correct the posterior slope, the varus-valgus mechanical axis and the resection height.

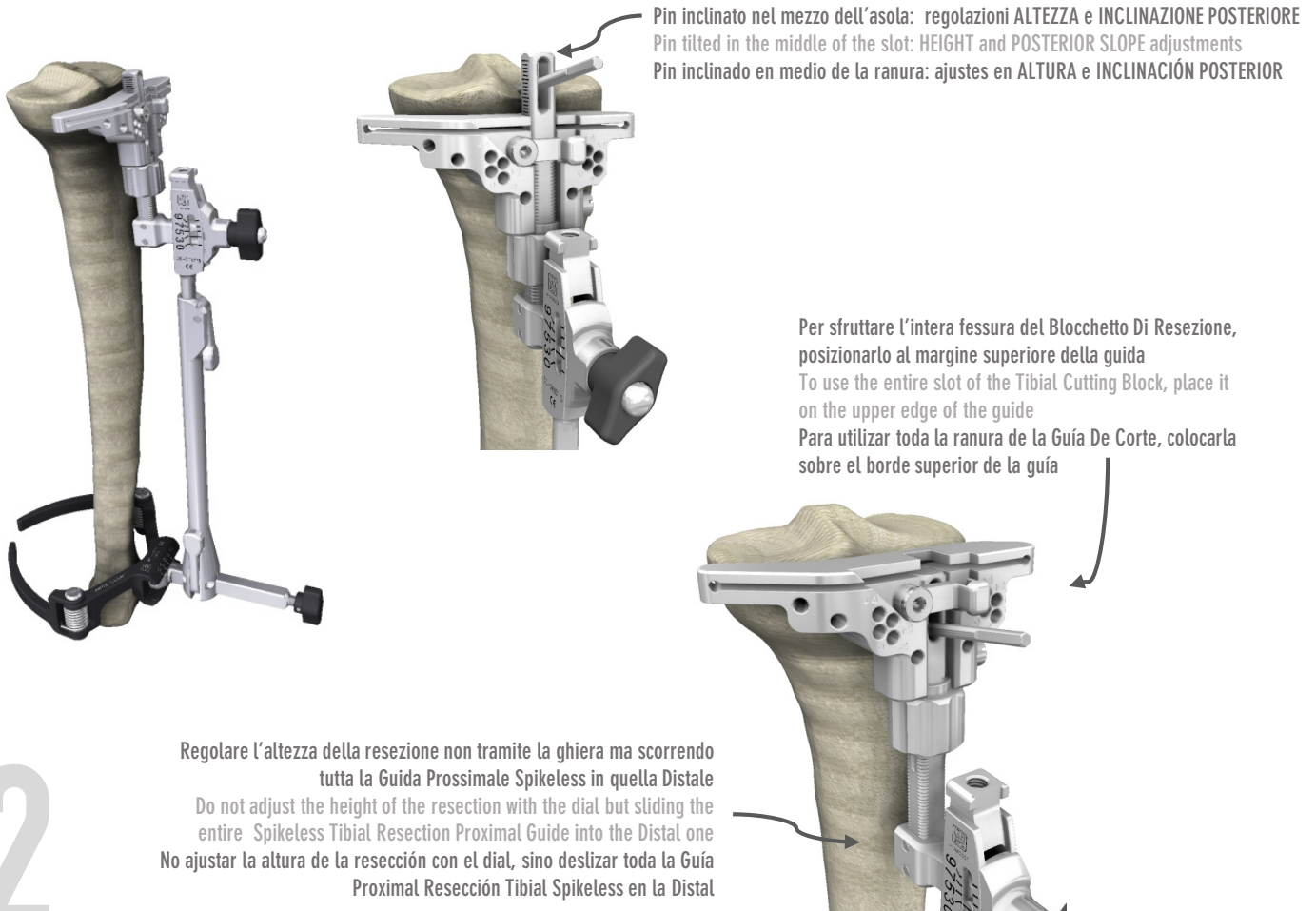
The locking pin may also be inserted in the slot of the Tibial Cutting Block.

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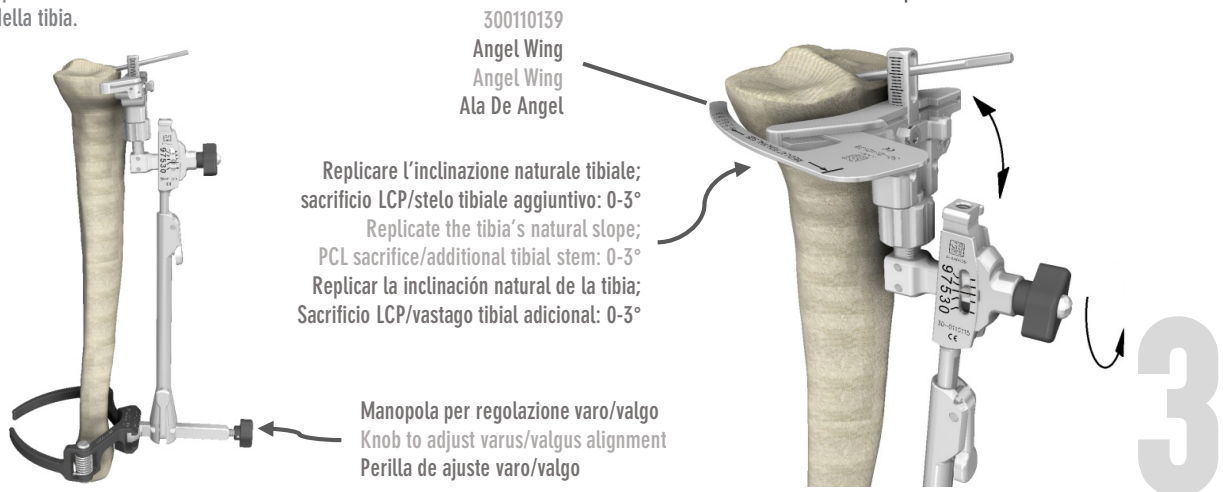
The alignment of the frontal mechanical axis in the varus-valgus plane may be adjusted by unscrewing the knob on the Tibial Distal Guide and sliding the guide on the Ankle Clamp in the ML plane.

The posterior slope may be adjusted by using the Angel Wing (300110139) and operating the knob on the Spikeless Tibial Resection Proximal Guide and/or the AP trigger on the Tibial Distal Guide. In general, proximal tibial resection should follow the tibia's natural posterior slope.

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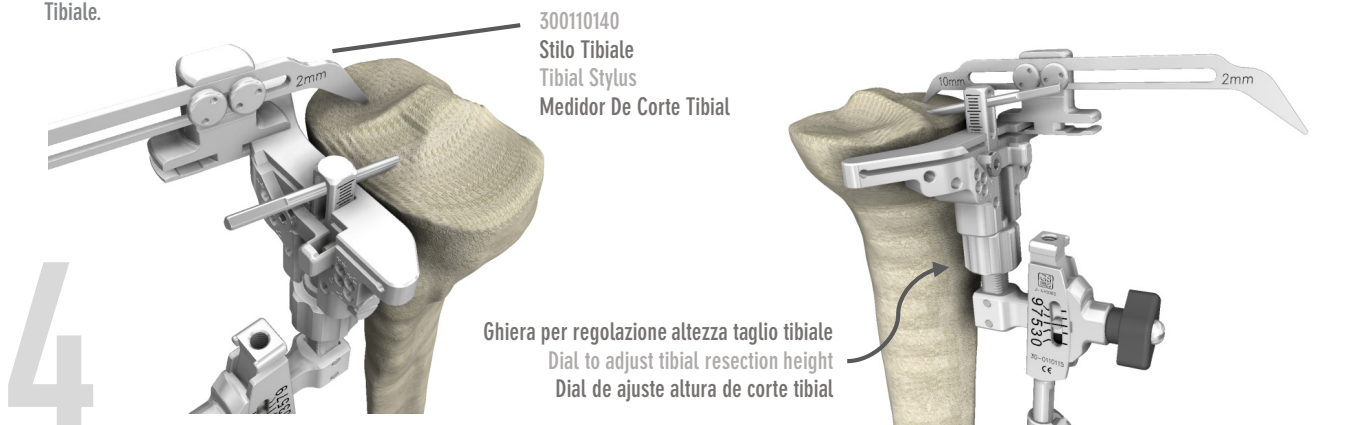
Position the Tibial Stylus (300110140) in the slot on the Cutting Block. The surgeon may choose to use as a cutting reference the lowest point on the most worn side of the tibial plateau and carry out a minimum resection (2mm tip for 2mm resection) or the highest point on the least worn side (10mm tip for 10mm resection). Check that the cut is adequate and if necessary modify the size of the resection by operating the dial and making a micrometric vertical shift of the Tibial Cutting Block.

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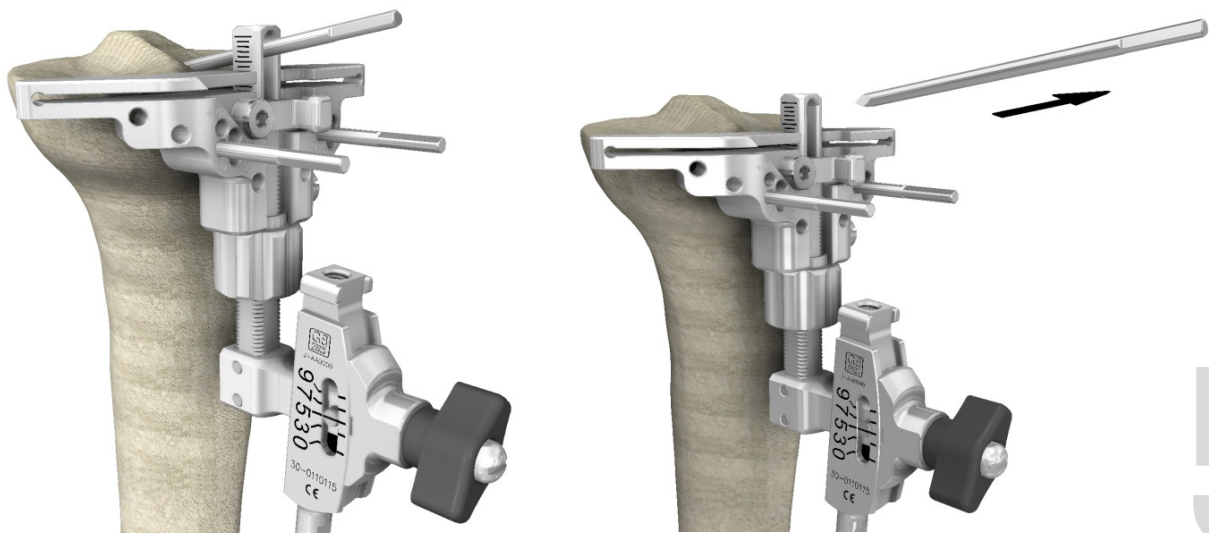
Anchor the Cutting Block with two Pin 3,2mm x 80mm (300110144) in the parallel 0 rows of holes.
Remove the temporary pin from the slot on the Spikeless Tibial Resection Proximal Guide.

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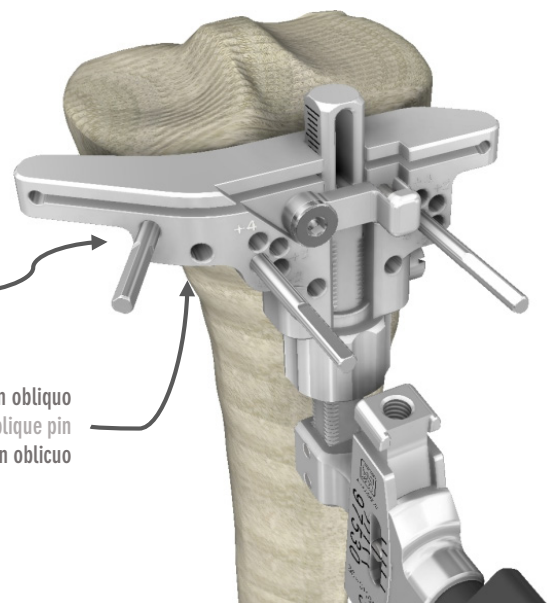
The guide can be made more stable by inserting a third pin in an oblique hole on the medial side of the Cutting Block. Such pin may be either the Pin 3,2mm x 80mm (300110144), the Pin with stop 3,2mm x 40mm (300110198) or the Threaded pin Ø3,2mm- Ø4,2mm (300110379).

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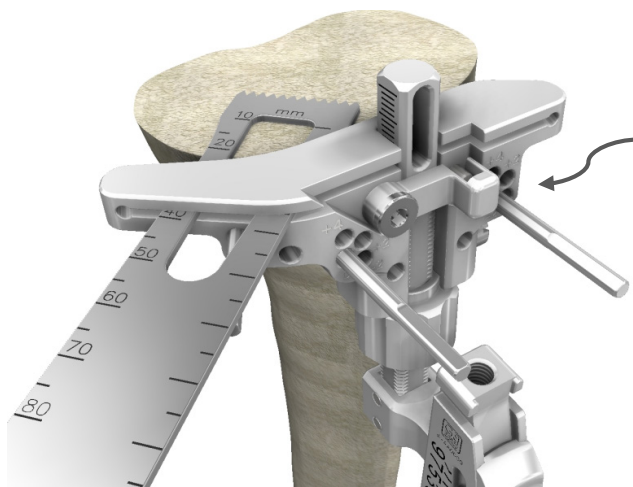
Make sure that the trigger on the Cutting Block is properly closed and perform the resection of the proximal tibia keeping the extra-medullary tibial resection guide in place. Use the Angel Wing to check the planarity of the cut and, if necessary, refinish the cut.

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If an oblique pin has been inserted, remove it. Raise the trigger on the Cutting Block and slide the dial downwards. Remove the extra-medullary tibial resection guide. Remove the Cutting Block by sliding it outwards. Leave the two parallel pins in for the next steps.

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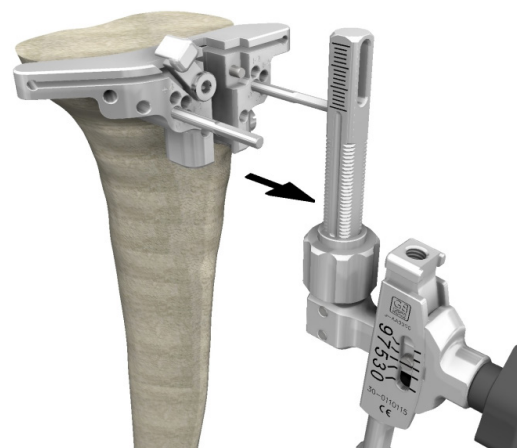
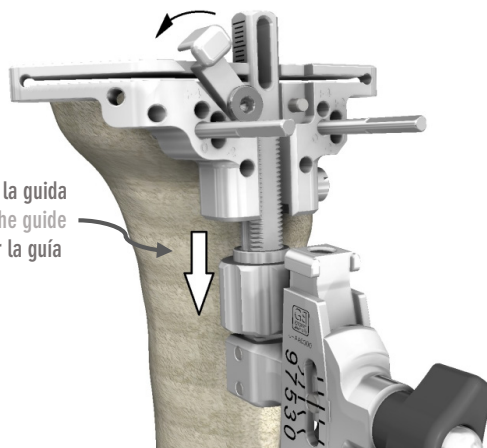
Use the Femoral IM Reamer 8.25 mm (300110145) to prepare the intramedullary hole in the femoral canal. The drill hole must be located approximately 1 cm above the femoral attachment of the posterior cruciate ligament and slightly medial to the mid-line of the femur. Removal of osteophytes from the margins of the intercondylar notch may aid identification of landmarks.

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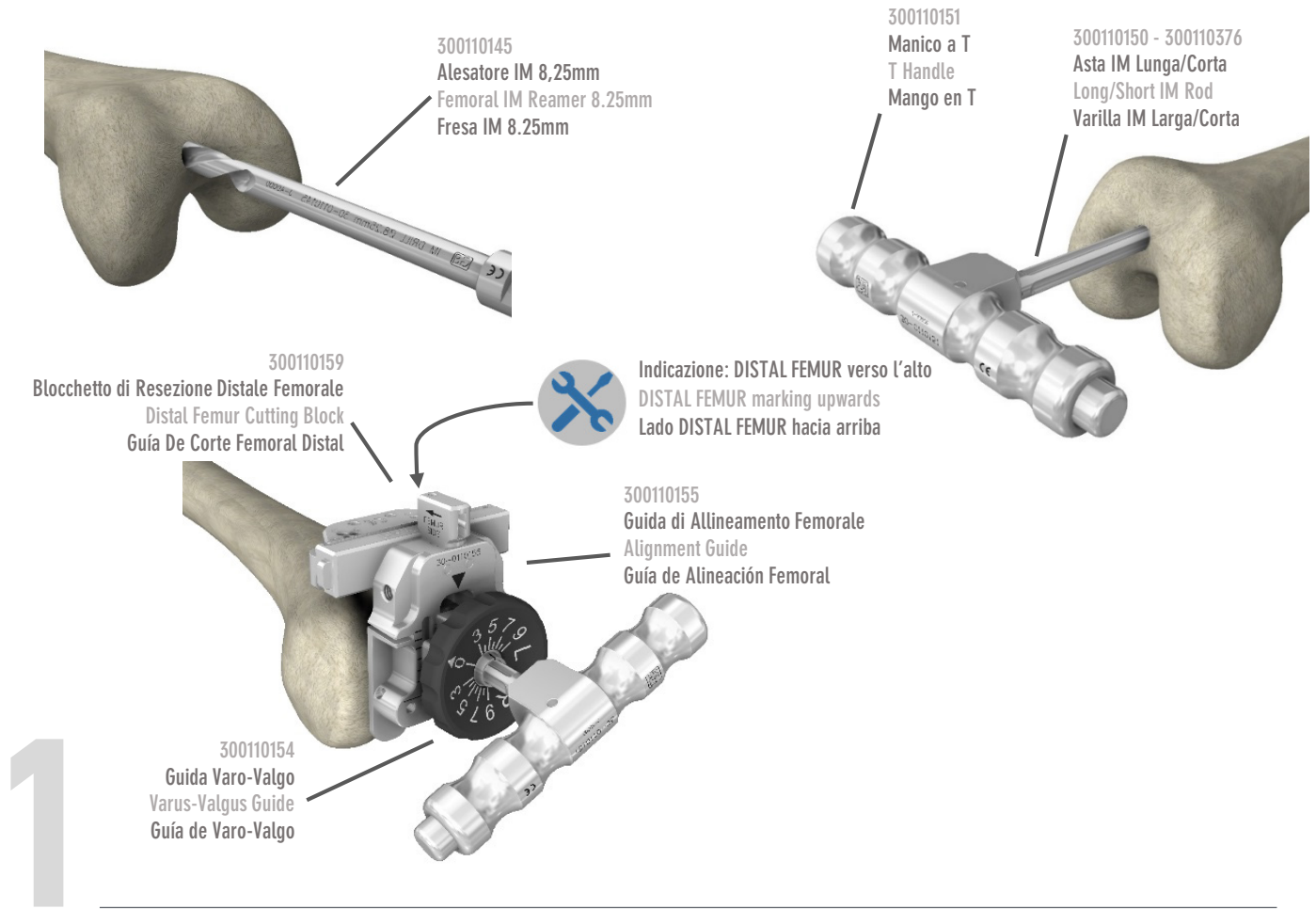
Assemble the Alignment Guide (300110155), the Distal Femur Cutting Block (300110159), with the DISTAL FEMUR marking visible on top and the Varus-Valgus Guide (300110154), with the FRONT marking visible on top.

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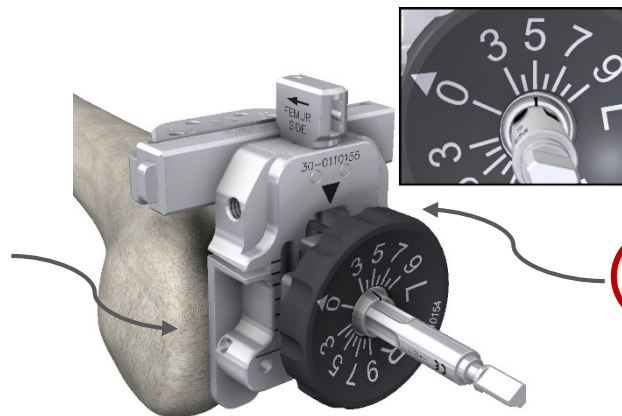
Preselect the valgus angle value determined in pre-operative planning: turn the bushing so that the marking relative to the leg being operated on, L (Left) or R (Right), is visible on the side facing upwards and set the desired valgus angle (range: 0° - 9°) by positioning the relative marking by the sign on the hole through the Varus-Valgus Guide. Insert the Distal Femur Cutting Guide on the IM Rod making sure that the Alignment Guide is in contact with at least one of the two femoral condyles.

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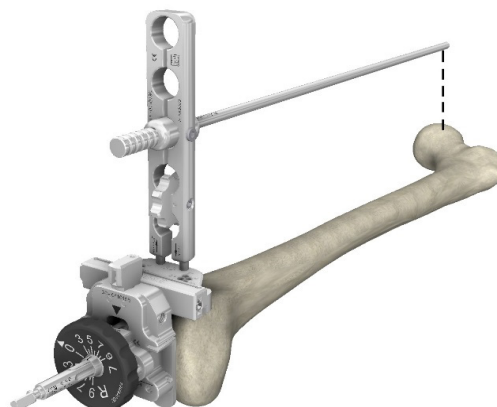
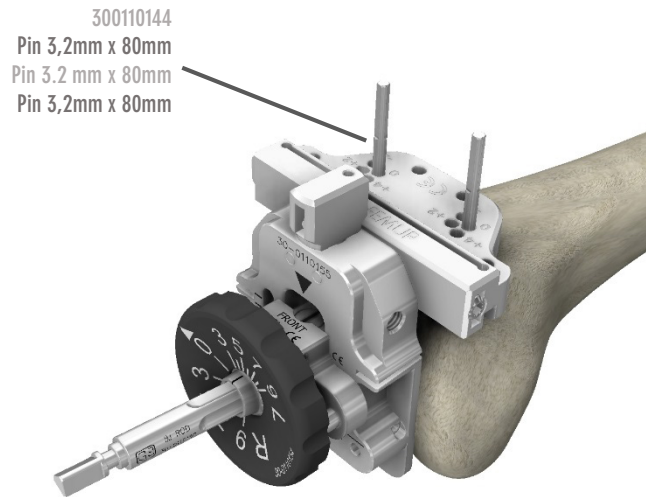
Fix the Distal Femur Cutting Block with two Pin 3.2 mm x 80mm (300110144) in the parallel rows of 0 holes.

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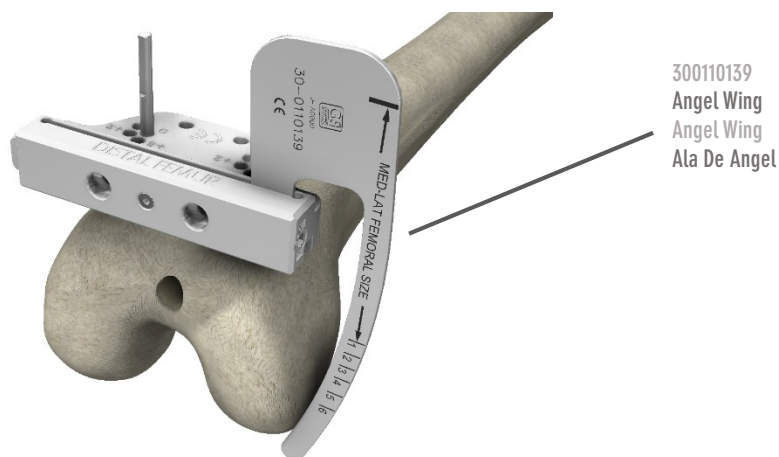
Release the Guides by pressing the button on the side of the Distal Femur Cutting Block and slide them off the IM Rod. Remove the IM Rod from the femoral canal using the relative T-Handle.

Check the position of the resection plane using the Angel Wing (300110139). If desired, 2 mm or 4 mm of additional bone may be resected by repositioning the Cutting Block over the Pins through the +2 or +4 holes respectively, or 2 mm less bone may be resected using the -2 holes.

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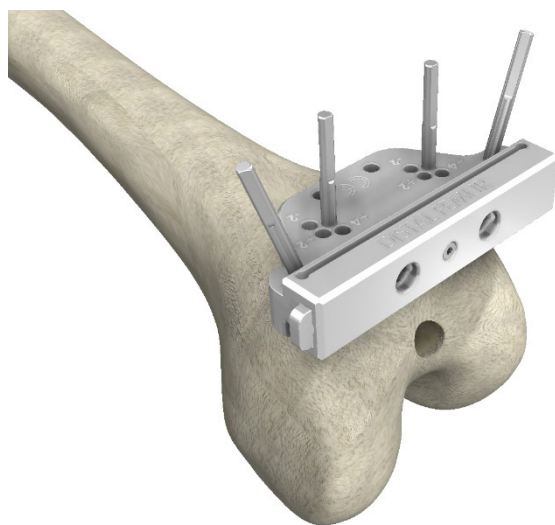
The stability of the Cutting Block can be improved by inserting one or two Pins in the oblique holes in the Cutting Block. The following pins are available: Pin 3.2mm x 80mm (300110144), Pin With Stop 3.2mm x 40mm (300110198), Threaded Pin Ø3.2mm- Ø4,2mm (300110379).

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Carry out the distal femoral resection. Check the planarity of the cut using the Angel Wing and if necessary refinish the cut.

If oblique pins have been inserted, remove them. Remove the Cutting Block by sliding it outwards. Leave the two parallel Pins in for the next steps.

If necessary, a further resection of 2 mm or 4 mm may be made by repositioning the Cutting Block on the Pins using the +2 or +4 holes.

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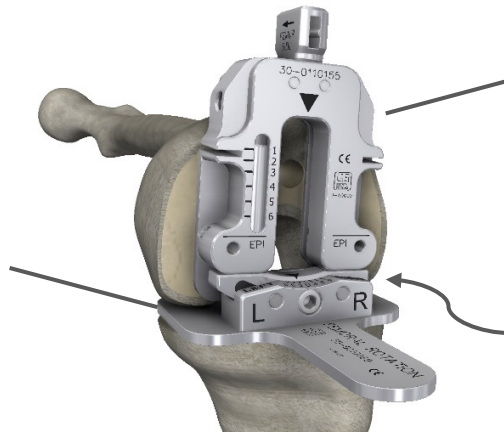
Assemble the Alignment Guide (300110155) and the Extra-Rotation Guide $\pm 9^\circ$ (300110169). It's possible to pre-select the femoral extra-rotation value (from 0° to 9°). Loosen the locking screw with the Hex Screwdriver For Screws 4,5-6,5mm (340085045), select the desired value with the tip of the arrow, check the L (Left) and R (Right) marks depending on the leg being operated, and then tighten the screw again. With the leg at about 90° flexion, put the assembled components into position and make sure that the Alignment Guide is completely in contact with the distal femoral cutting plane and that the adjustable Extra-Rotation Guide is in contact with the posterior condyles.

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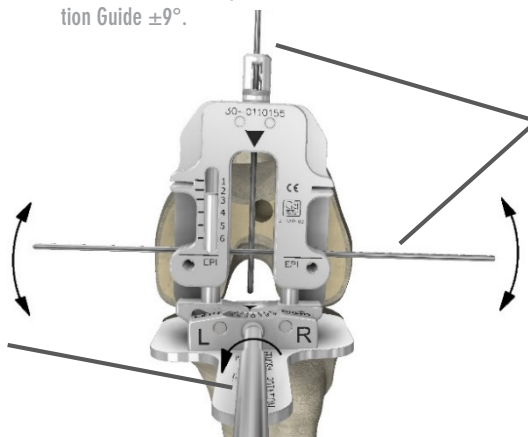
The surgeon can use one or two Guide Wire (300110175) to check the correct femoral extra-rotation, aligning them with the chosen landmarks as Whiteside's line or the Trans-Epicondylar axis, or both. If necessary, free the Extra-Rotation Guide $\pm 9^\circ$ by loosening the locking screw with the Screwdriver and manually rotating the Alignment Guide till the correct alignment is obtained. Then tighten the locking screw to fix the final position of the Extra-Rotation Guide $\pm 9^\circ$.

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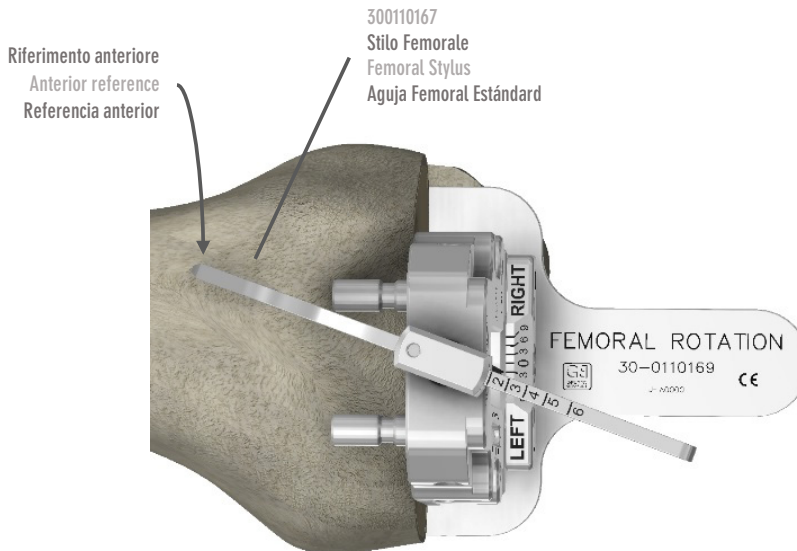
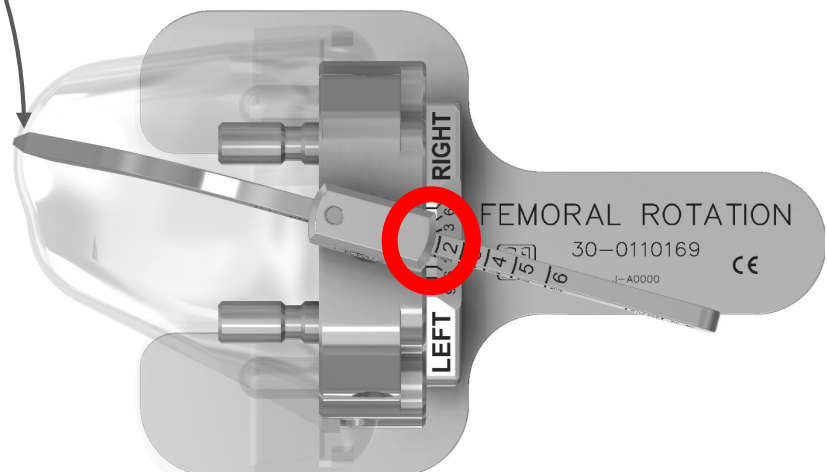
Insert the Femoral Stylus (300110167) in the Alignment Guide as per the FEMUR SIDE indication, which will point in the direction of the femur. Set the Femoral Stylus to the size measured during the preoperative planning, sliding it through its support and positioning the Femoral Stylus tip on the anterior cortex of the femur at the desired run-out point of the anterior resection. The preselected point represents the anterior reference and must therefore be positioned preferably towards the highest lateral part in order to avoid the problem of anterior femoral notching.

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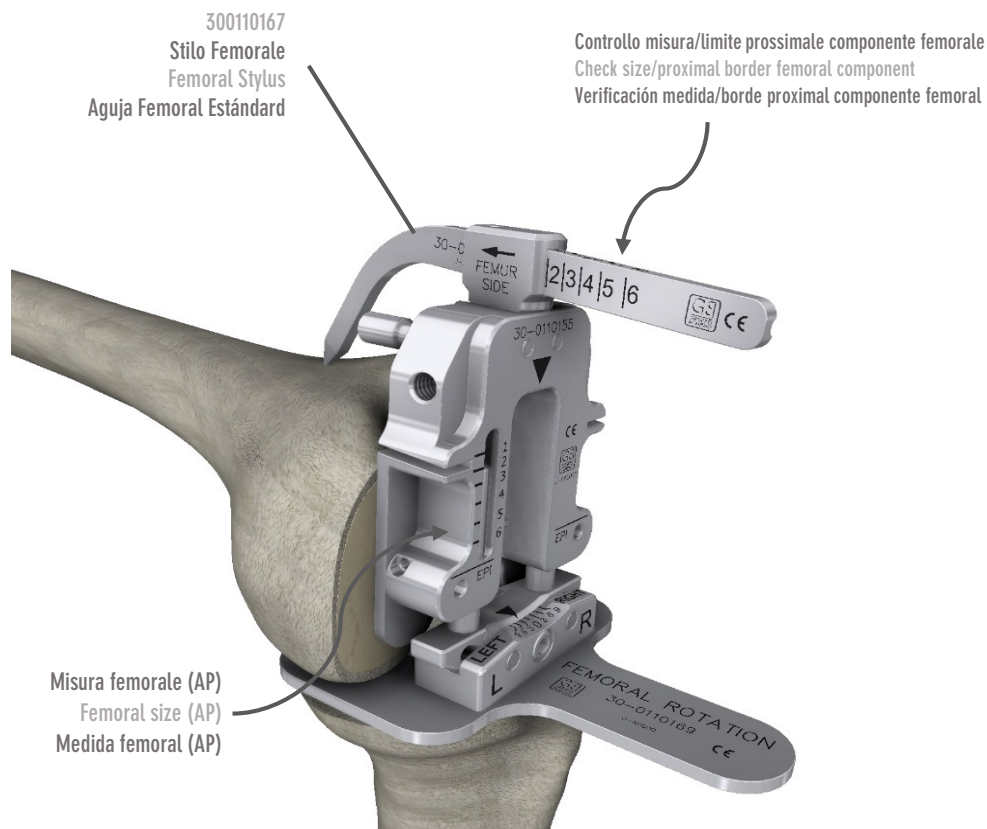
Read the femoral size through the frontal slot of the Alignment Guide. If it doesn't correspond to the stylus setting, reset the stylus and re-check the size.

If the indicator is midway between two sizes, the surgeon may choose the smaller or the bigger, because the TOUCH procedure is anterior referencing, so the anterior cut is in the same position for every size of femoral component. If the surgeon chooses the bigger size in order to keep more posterior condylar bone, the flexion gap is narrowed and there is a risk of excessive ligament tension and consequent reduction of range of motion. If the surgeon chooses the smaller one, the range of motion will probably be better but there will be a risk of having more laxity in flexion.

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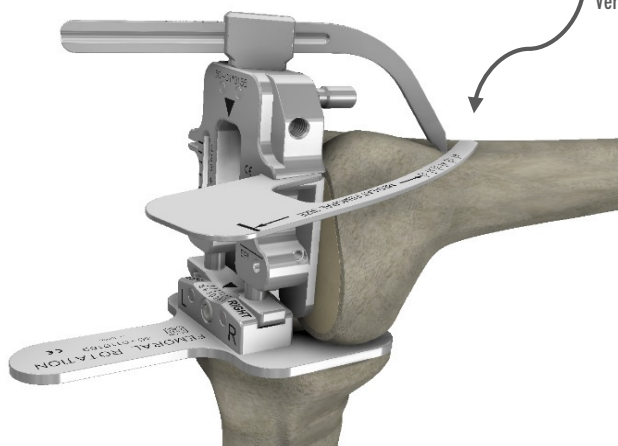
To check the position of the anterior resection insert the Angel Wing in the lateral or medial slot on the Alignment Guide.

To check the position of the anterior resection insert the Angel Wing in the lateral or medial slot on the Alignment Guide.

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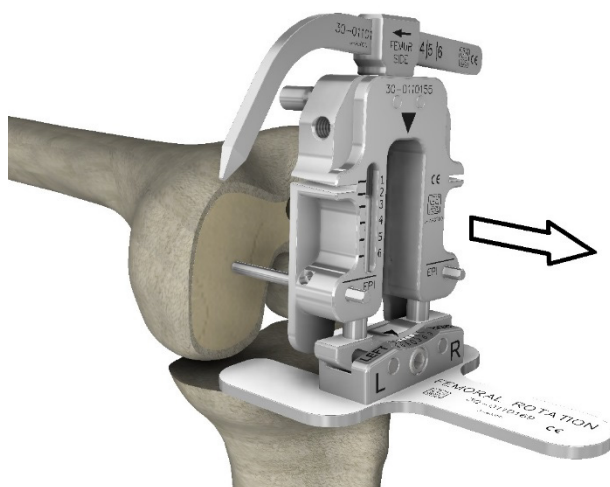
Insert two Pin 3.2mm x 80mm (300110144) in the holes in the Alignment Guide; rotate the Stylus to the medial side and remove the Guide outwards.

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The 4in1 Femoral Cutting Block (300110176÷81) has an outside measurement in the ML plane equal to that of the relative femoral component. Check for correspondence between the size chosen in AP and its measurement in ML. Should it be necessary to scale the femoral component up or down, see the recommendation in 2 above.

Should the size in AP be adequate but the bulk in the ML plane be excessive, for sizes 2, 3 and 4 it's possible in the final phase of the implant to use the Skinny version (sizes 2-, 3-, 4-):

4in1 Femoral Cutting Block and articular geometry: same as the reference size;

Measurement in AP: equal to the reference size;

Measurement in ML: equal to that of the next size down, equivalent to a symmetrical reduction of 2 mm on both sides.

For example, Skinny size 3- has an outside measurement in AP equal to size 3 but an outside measurement in ML equal to size 2. To implant this, the surgeon should use the 4in1 Femoral Cutting Block and the size 3 Trial Femoral.

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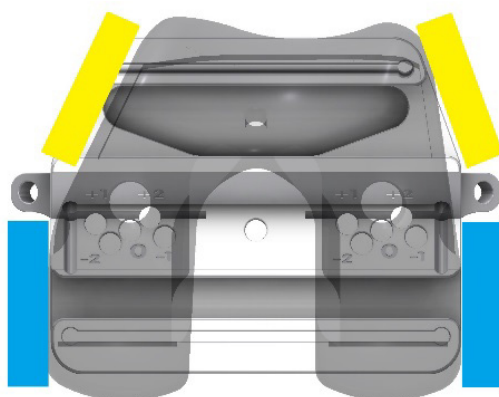
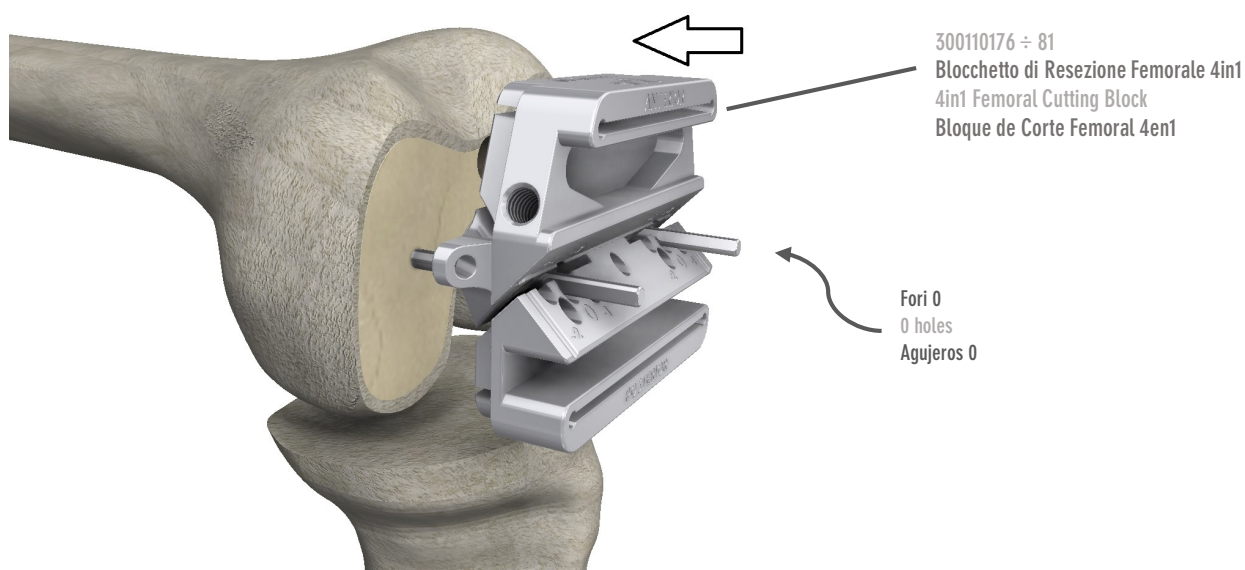
The position of the two Pins in the ML plane, and thus the position of the 4in1 Femoral Cutting Block, does not restrict the final positioning of the definitive component in that plane.

Place the 4in1 Cutting Block corresponding to the chosen size on the Pins using the 0 holes and slide it along until fully up against the distal resection plane of the femur.

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Before proceeding with the femoral resections, carry out the following checks.

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Check the position of the anterior resection with the aid of the Angel Wing. To avoid the problem of anterior femoral notching, the exit angle of the anterior resection is 4°.

If necessary, correct the level of the anterior resection by moving it 1 mm or 2 mm posteriorly or anteriorly by repositioning the Cutting Block in the holes marked +1, +2 or -1, -2.

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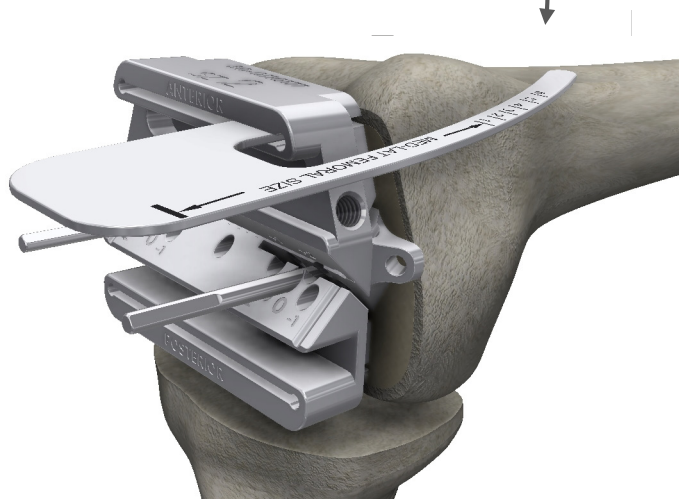
FLEXION GAP

Check the gap and the ligament balancing in flexion using the Spacer Block of the thickness determined for the extension gap. Inserted from the side marked FLEX, it must be flat up against the tibial resection and the lower part of the 4in1 Femoral Cutting Block. The Spacer Block and the Cutting Block reproduce the total thickness of the prosthesis in flexion.

If necessary, the width of the gap in flexion may be modified by:

Moving the Cutting Block 1mm or 2mm posteriorly or anteriorly into the holes marked +1, +2 or -1, -2. Important: in this case the anterior reference is modified!

Changing the size of the femoral component. The difference in AP between two different femoral sizes is 4 mm, whereas between sizes 5 and 6 it is 5 mm. Scaling the femoral component up or down does not alter the anterior reference, and the recommendations in 2 above apply here too.

If the contact between Cutting and Spacer Blocks is not uniform, it's possible to create a trapezoidal instead of rectangular flexion gap. In this case, consider a ligament correction and/or a different femoral extra-rotation setting.

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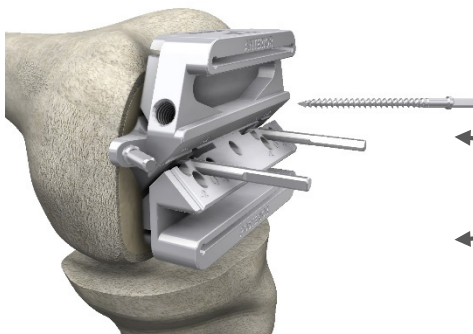
Fix the 4in1 Femoral Cutting Block in the final position by putting two Threaded Pin Ø3.2mm - Ø4,2mm (300110379) or two Pin With Stop 3.2mm x 40mm (300110198) in the oblique holes.

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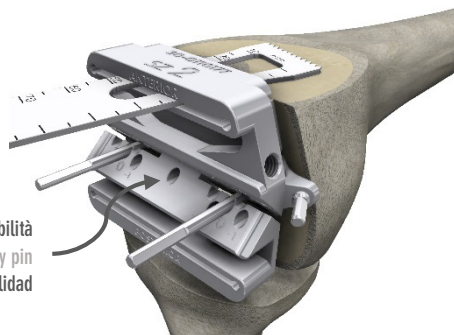
For greater stability it's possible to put a Pin in the central hole of the Cutting Block and/or apply the Universal Handle (300110279) and Universal Handle/Pin Extractor (300110361). Remove the Pins and the 4in1 Femoral Cutting Block.

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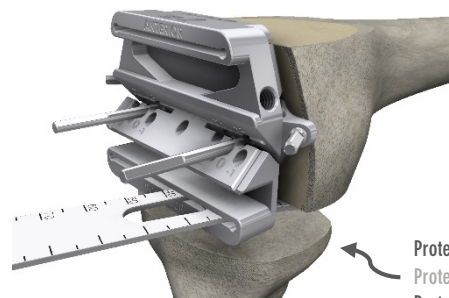
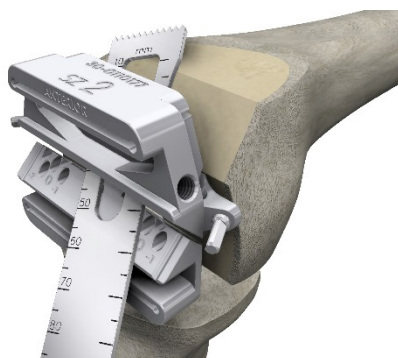
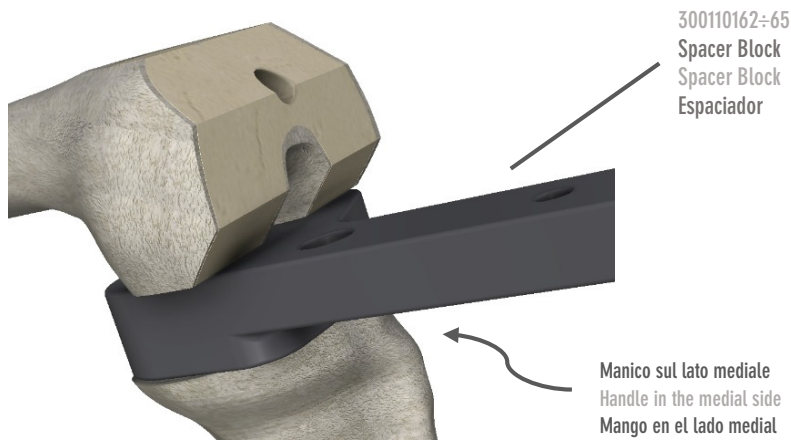
Check there is the right gap and correct balancing of the tension of the ligaments in flexion using the Spacer Block defined for the extension gap.
 The Spacer Block must be used from the side marked EXT and is asymmetrical to limit interference with the patellar ligament.

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Position the femoral box guide K-MOD PS Fem Box Guide (30011101÷106) for the chosen size on the anterior, distal and anterior oblique femoral planes. Place it in a central position, or slightly lateral, on the ML plane. The medial side of the guide goes on the medial side of the definitive component. In the case of Skinny femoral components (2-, 3-, 4-), the size is indicated on them.

Make sure there is perfect adherence between the osseous beds and the metal surfaces of the guide.

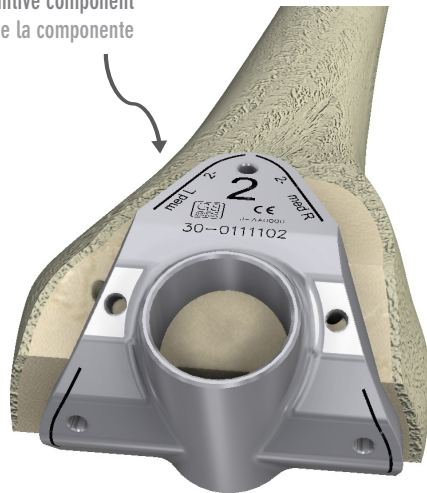
Fixate the guide using three pins, two of which positioned in the holes on the distal and anterior oblique surfaces (medial and lateral) and one on the anterior surface. Use the Threaded pin Ø3.2mm- Ø4,2mm (300110379) or Pin With Stop 3.2mm x 40mm (300110198) or Headed Pins (300110197).

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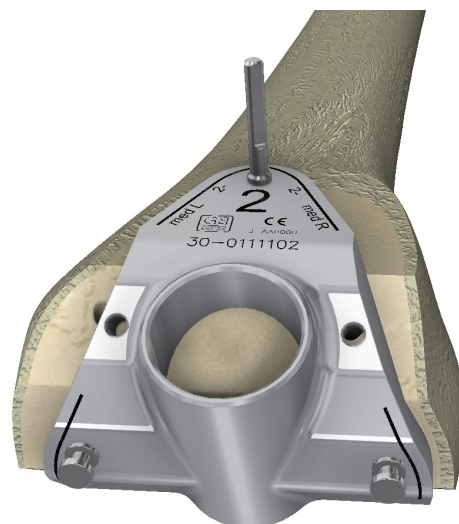
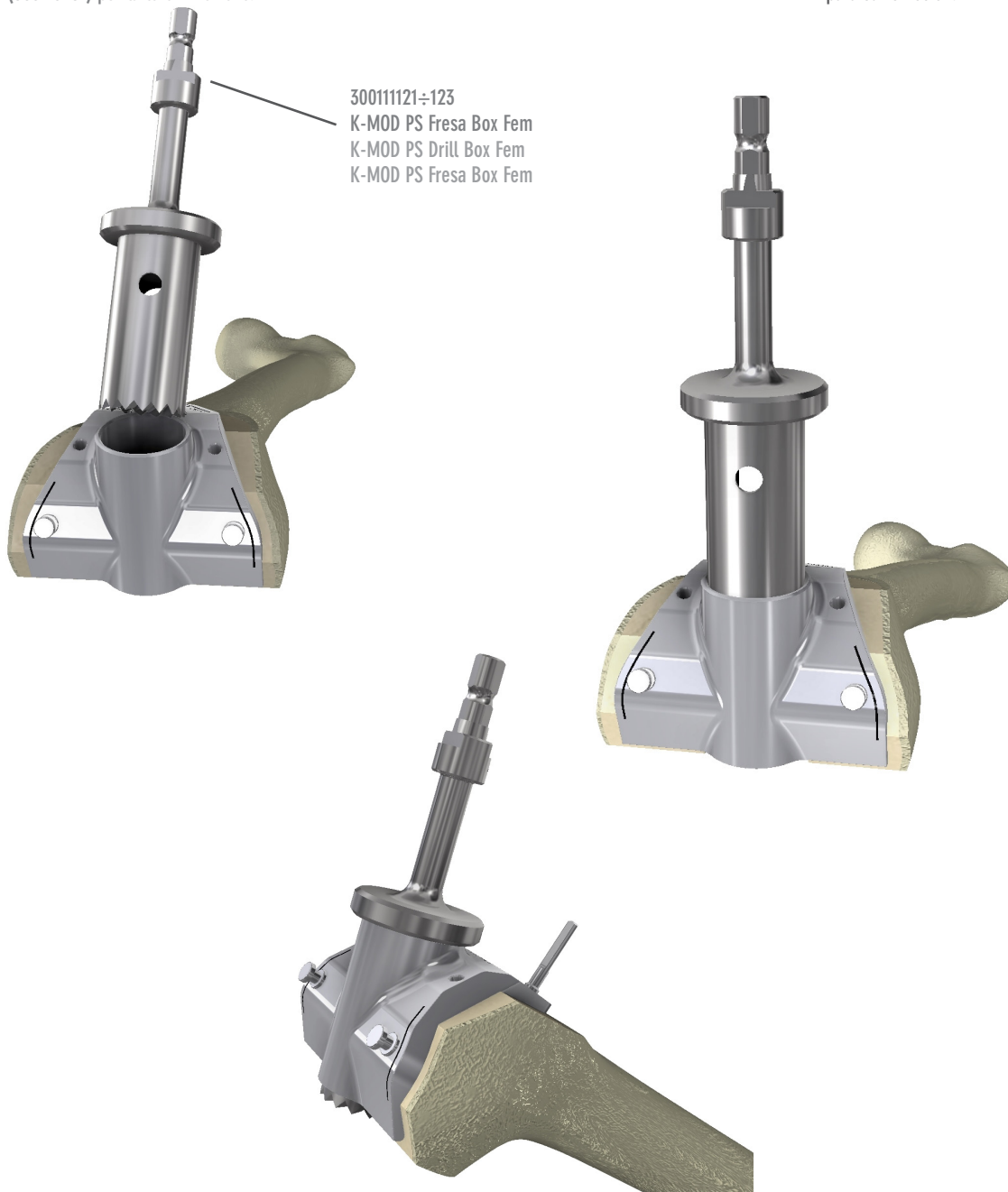
Select the reamer K-MOD PS Drill Box Fem (1-2, 3-4, 5-6: 300111121÷123) for the chosen femur size. Position it axially in the cylinder in the PS Fem Box Guide and produce the PS femoral box by advancing the reamer up to the stop.

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Remove the reamer and PS Fem Box Guide by extracting the Pins and. If Headed Pins are being used, use the Universal Handle/Pin Extractor to remove them.

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Assemble the Femoral Trial (300110223÷34; 300111201÷26) of the chosen size and for the side of the leg operated (L: Left; R: Right) on the Femoral Positioner (300110235).

Should it be necessary to use a Skinny component with a reduced measurement on the ML plane (sizes 2-, 3-, 4-), the Femoral Trial to use will in any case be the reference size (2, 3, 4). On turning the central nut, the hooks close (LOCK) and open (RELEASE) to hold or release the component.

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Position the Femoral Trial on the femur; the recommended position on the ML plane is central or slightly lateral to facilitate patellar slide; in case of PS femoral component, refer to the cylindrical slot made with the Drill Box. Avoid protrusion of the component with respect to the bone.

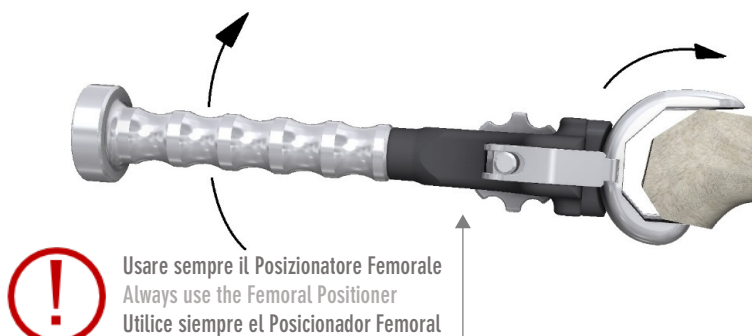
The ML measurement of the Skinny femoral components is symmetrically reduced by around 2 mm on both sides compared to the reference size Trial.

Depending on the inclination of the anterior resection, impact making sure the handle of the Femoral Positioner is turned upwards, so as to maintain congruence between the resections and the Trial's internal surfaces.

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Release the Positioner and complete the positioning with the Femoral Impactor (300110259). Make sure the Trial is perfectly adherent to the resections and check for any medial-lateral protrusion.

NB.: it is also possible to insert the Femoral Trial after positioning the tibia trial components. In this case, position it in deep flexion and impact with the leg at around 90° flexion.

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Skinny size: consider 2mm reduction on both sides ML

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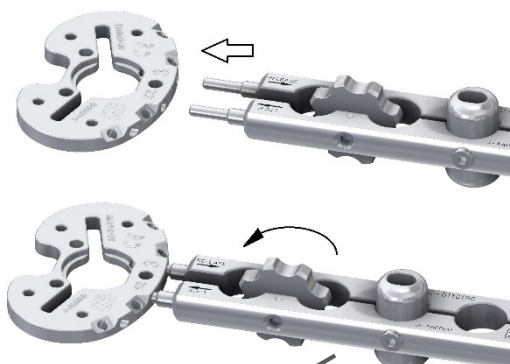
Using the Tibial Baseplate (300110190÷95), select the tibial size that provides the desired tibial coverage.

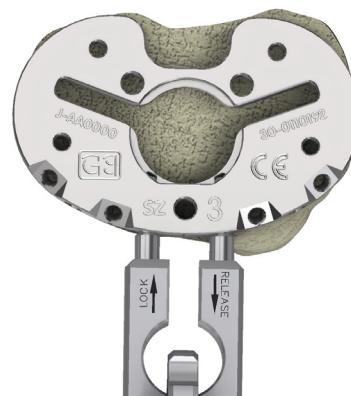
The Baseplate can be assembled onto Tibial Baseplate Handle (300110196) by means of the two anterior holes; on rotating the central nut, the two pins of the Handle close (LOCK) or open (RELEASE) to hold or release the Baseplate.

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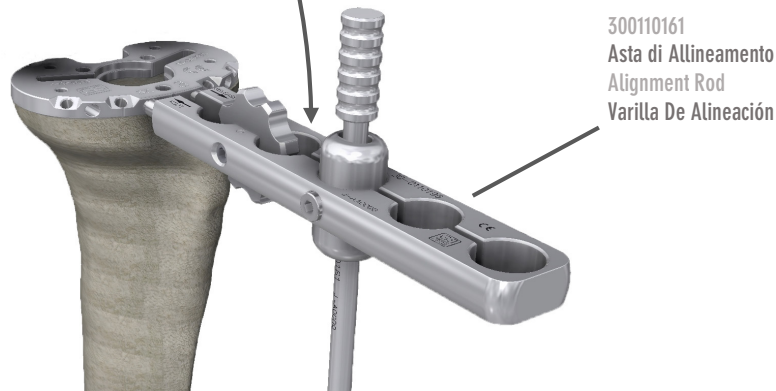
Correct positioning of the Baseplate and alignment with the tibial mechanical axis can be checked by inserting the Alignment Rod (300110161) in the Handle. Tighten the nut well to prevent the Rod from shifting in the varus/valgus plane.

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If desired, it is possible to fix the positioning of the Baseplate with Headed Pins (300110197) in the anterior oblique holes. These pins in any case make it possible to position the trial insert and correctly perform the trial reduction. They can be removed with the Universal Handle/Pin Extractor (300110281).

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300110197
Headed Pins
Headed Pins
Headed Pins

300110281
Manico Universale/Leva Pin
Universal Handle/Pin Extractor
Mango Universal/ Pin Extractor

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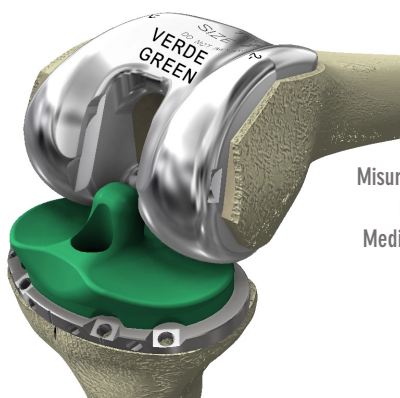
[K-MOD PS]

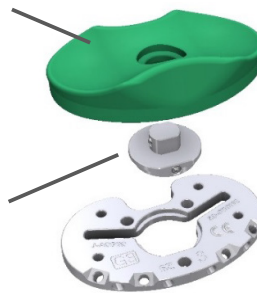
Misura inserto = femore
 Insert size = femur
 Medida inserto = femur

Release the Handle from the Baseplate and position the Trial Insert on the Baseplate by choosing one of the following options

- **FIXED INSERT:** posterior cruciate ligament retaining CR (300110441÷76), Ultra Congruent UC (300110481÷516), Dynamic Congruence DC (300110521÷96) depending on the side of the leg being operated (L: Left; R: Right) or Posterior Stabilized PS (300111301÷336). The Trial is applied by fitting the lower part into the special slot in the Baseplate.
- **MOBILE INSERT:** UltraCongruent UC (300110401÷436) or Posterior Stabilized PS (300111301÷336); put the Mobile Trial Insert Peg (300110395) in the Baseplate. The Trial hooks onto the Peg by means of a special hole in the bottom, thus remaining free to rotate.

NB.: If desired, it is possible to leave the Handle in during the trial (tibial sizes 1 and 2: **DO NOT** insert completely).

The size of the tibial insert corresponds to that of the femoral component, even if Skinny femoral components are subsequently used.

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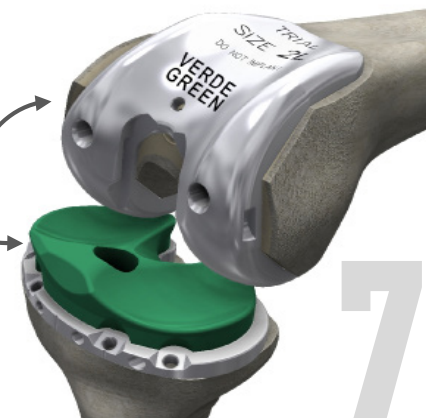
Assess the stability of the joint in extension and flexion. The extension check should be performed with the knee flexed a few degrees to relax the posterior capsule. However, the knee should be able to extend fully. The flexion check should be performed with the knee flexed to 90°.

Optimum stability is achieved when the medial and lateral opening is similar to that of a normal knee under valgus and varus stress. An adjustment of ligament balance may be needed if there is differential ligament tightness between varus and valgus in flexion or extension.

After achieving the correct rotational position of the tibial plate and making sure that does not protrude beyond the tibial profile, the surgeon can mark the tibia by the frontal marks. This will be the landmark for repositioning the Baseplate and preparing the bed for the tibial stem. The Baseplate can also be fixated using the pins in the anterior oblique holes.

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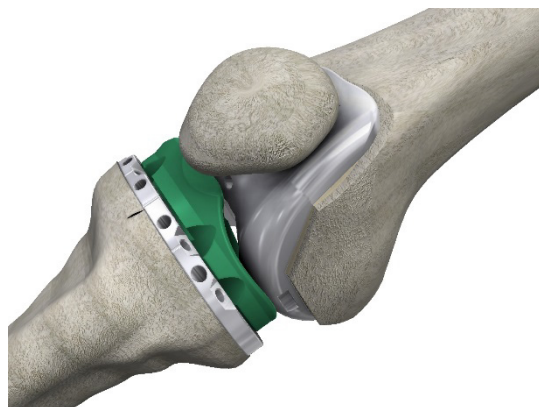
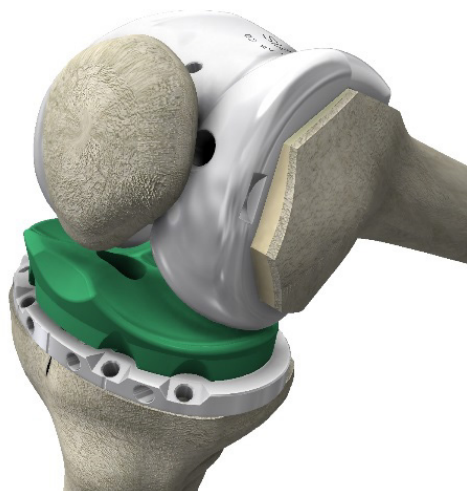
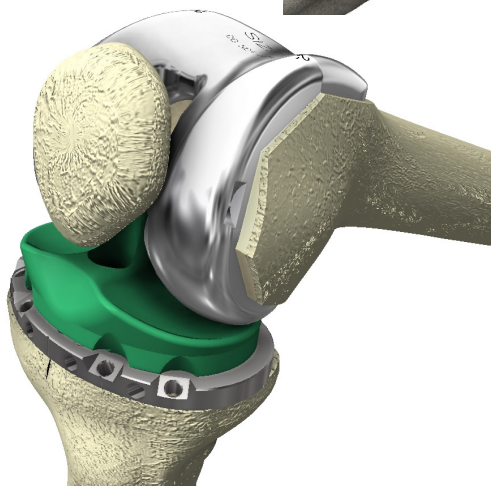
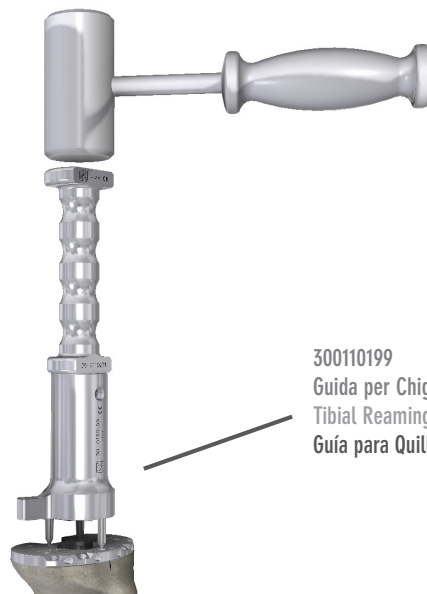
Remove the Trial insert and, in the case of a mobile insert, also the Mobile Trial Insert Peg (300110395). Insert the Tibial Reaming Broaching Keel Guide (300110199) in the Baseplate by impacting the three spikes into the proximal tibia through the holes in the Baseplate. For impacting, it's possible to insert the Tibial Broach Handle (300110271) in the Keel Guide.

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Prepare the conical slot in the bone using the Conical Tibial Reamer (300110200) and inserting it into the Guide as far as it will go.

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To prepare the slot for the keel, select the
Keel Broach (Sizes 1-2: 300110254, Sizes 3-4:
300110255, Sizes 5-6: 300110256) correspon-
ding to the size of the chosen tibial Baseplate.
Assemble the Broach on the Broach Handle
(300110271) and impact it through the Guide as
far as it will go.

To limit the risk of fracturing of the tibial tray
in the case of particularly sclerotic bone, it is
best to prepare the slots for the fins using a
small osteotome.

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Valid procedure for femoral component NOT posterior stabilized PS

With the Femoral Trial properly positioned, make the housing for the two femoral pegs by inserting the Femoral Peg Drill (300110246) in the two holes on the Trial and proceed as far as the stop.

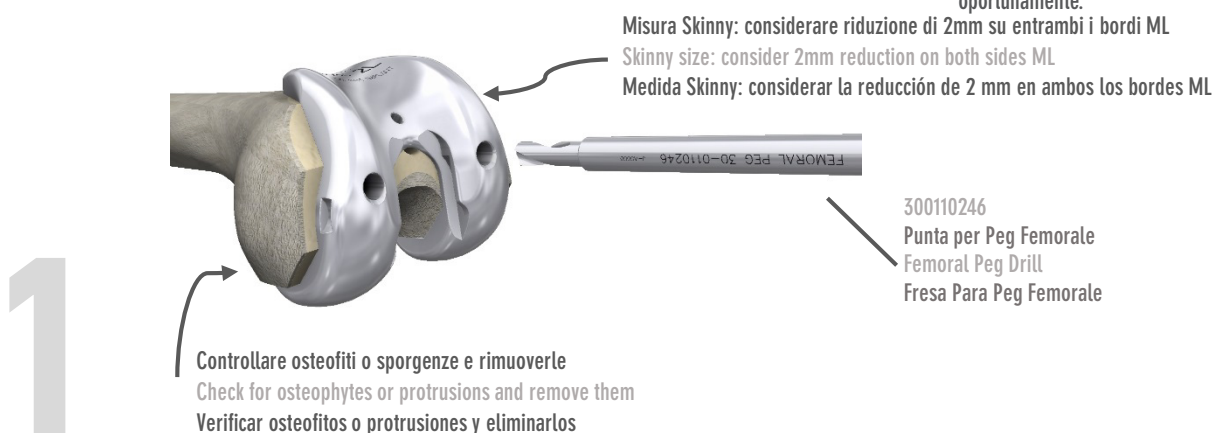
NB.: if Skinny femoral components (2-, 3-, 4-) are subsequently to be used, use the reference size Trial (2, 3, 4). When positioning it, bear in mind that the outside measurement of the Skinny component in the ML plane will be symmetrically smaller by around 2mm on both sides. It's advisable to check for the presence of osteophytes or protrusions on the posterior condyles with respect to the profile of the femoral component; such features could limit flexion and should be removed.

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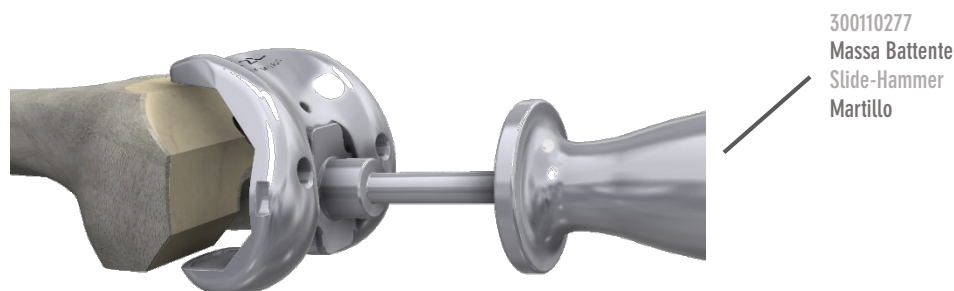
Remove the Trial Femoral using the Slide-Hammer (300110277), which has a plate on one end for coupling with the Trial itself. The narrower end of the plate is recommended for sizes 1 and 2, while the wider end is for sizes 3 to 6.

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Choose the definitive components with great care. The colour code on the packaging of the femoral components and inserts refers to the colour of the Trial Insert defined during the trial reduction.

The size of the polyethylene insert corresponds to that of the femoral component, even when Skinny femoral components are used.

The package of the fixed tibial component includes the locking screw of the polyethylene insert.

For both fixed and mobile tibial components, it is advisable to always check the effective locking of the tibial stem closure cap using the Hex Screwdriver For Screws 4,5-6,5mm (340085045).

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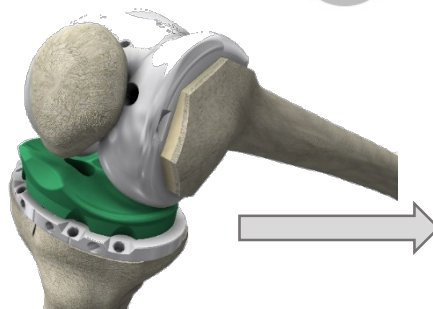
Color code - trial insert: definitive insert and femoral component

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Fixed tibia: locking screw included in the package

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When using cemented components, follow the reference cementing technique for the operating surgeon. Then carefully apply the cement. Thoroughly clean the surfaces that will house the implants, eg. with pulse lavage, and position the knee in deep flexion.

Luxate the tibia anteriorly and insert the tibial component using as reference points the marks already made and the preparation carried out for the keel.

Complete the positioning using the Tibial Tray Impactor (300110258) and make sure there is perfect adherence between the component and the cut.

When using cemented components, carefully remove all excess cement, especially in the posterior zone.

Thoroughly clean the upper surface before proceeding with the polyethylene implant.

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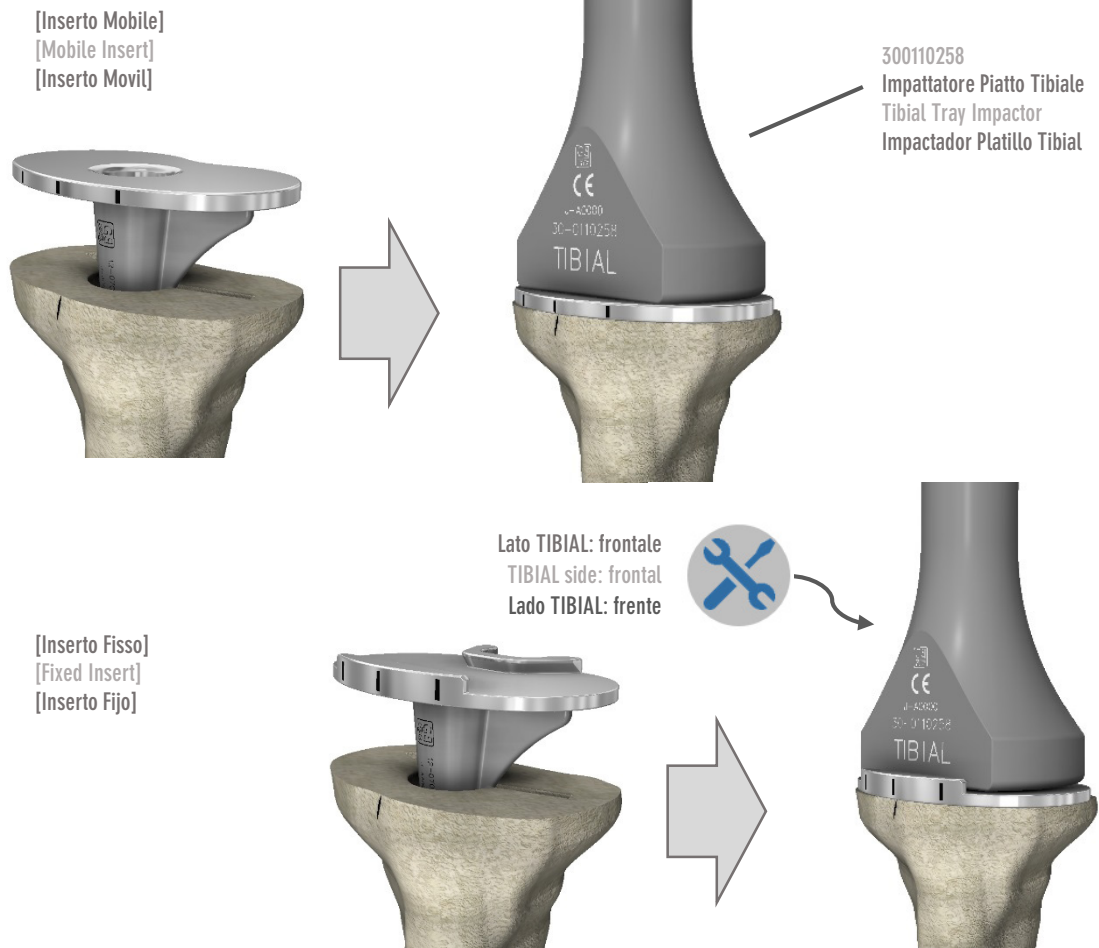
[Inserto Mobile]
[Mobile Insert]
[Inserto Movil]

MOBILE INSERT

Insert the rotating implant in the tibial tray. The ANT indication must be on the front side of the tibia.

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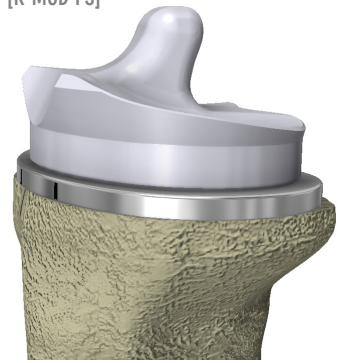
FIXED INSERT

Introduce the insert into the tibial tray by engaging the posterior side first, and keeping it inclined. The groove on the posterior edge of the insert must fit into the posterior profile of the tibial tray.

Now bring the anterior part into contact as well and apply force obliquely on the front edge of the insert so it can hook into the anterior profile of the tibial tray.

Using the Screwdriver (340085045), fully tighten the safety screw for micro-movement presents in the tibial component pack.

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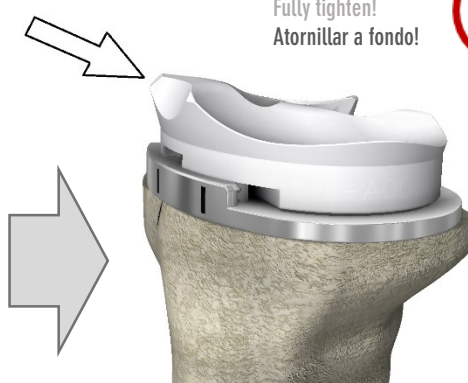
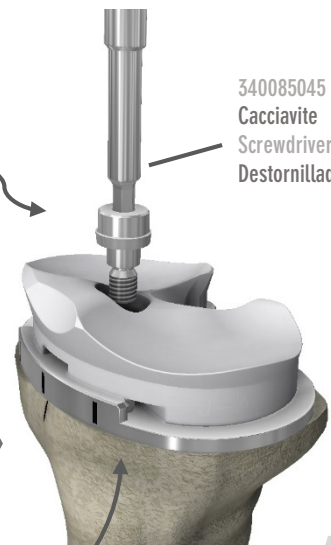
FIXED INSERT: SAFETY LOCKING SCREW

It's possible to tighten the safety screw for micro-movement AFTER implanting the femoral component. This makes it possible to check that the chosen thickness is adequate. Make sure that the hole for the screw is completely unobstructed and clean before starting to insert the screw.

Should it be necessary to change the insert AFTER anchoring it with the safety screw, unscrew it as far as the exit from the tibial plateau. Warning: the safety screw will in any case remain housed in the hole of the polyethylene insert! Remove the insert by leveraging upwards on the anterior wall and replacing it with a new one. Secure it using the screw in the single packet K-MOD Lock Screw x Fixed Insert M5 (120700701), which can be used with any size because micro-movement safety screws for K-MOD fixed tibial components are all the same.

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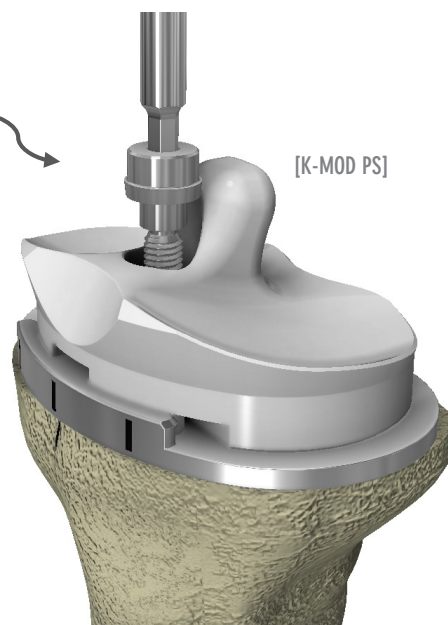
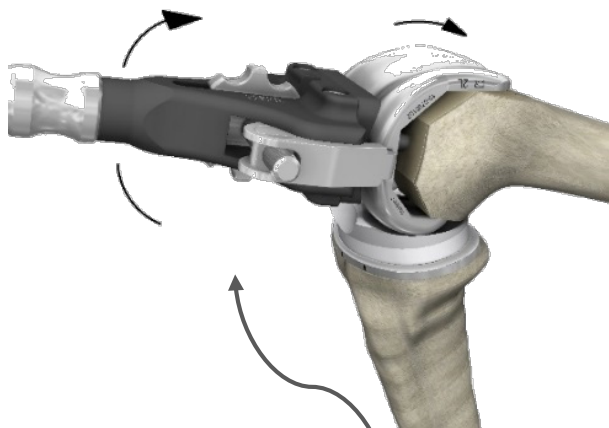
Assemble the femoral component on the Femoral Positioner (300110235). Rotating the central nut, the hooks open (RELEASE) and close (LOCK), to release or lock the component.

Put the knee in deep flexion. Position the femoral component on the femur starting from the posterior edge in contact with the posterior resections and making sure that the two pegs fit into the relative holes. Impact with the handle of the Femoral Positioner turned upwards, thus rotating the component from posterior to anterior, to obtain congruence between the resections and its internal surfaces.

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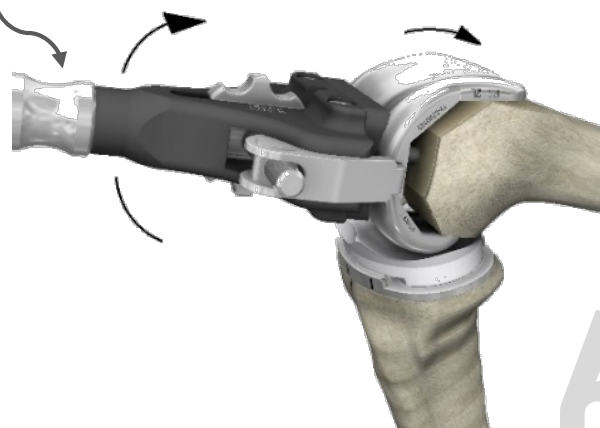
Now adjust the joint to 90° flexion. Disconnect the Positioner and complete positioning with the Femoral Impactor (300110259). Check there is perfect adherence between components and cuts.
When using cemented components, carefully remove all the excess cement.

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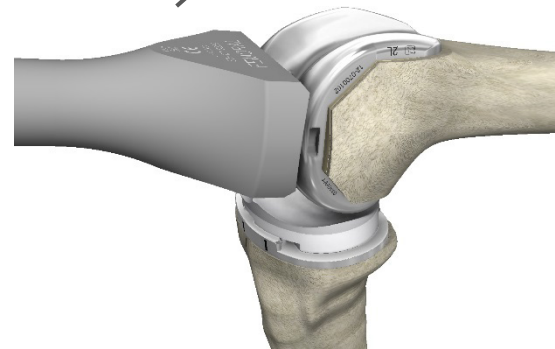
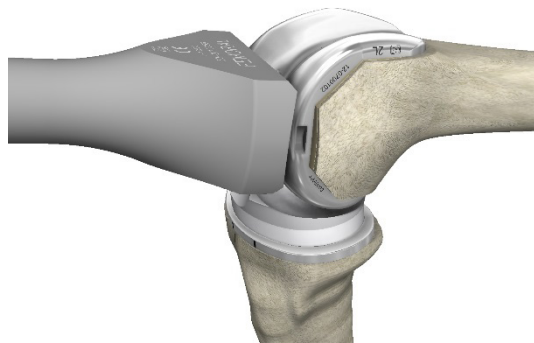
Settle the components by fully extending the leg.
When using cemented components, leave the leg in extension during the cement setting phase. Check and then proceed to remove any excess cement.

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Assess knee movement and stability within the Range of Motion.
Thoroughly irrigate and clean the joint of all residues.
Proceed with suturing and closure in the surgeon's preferred manner.

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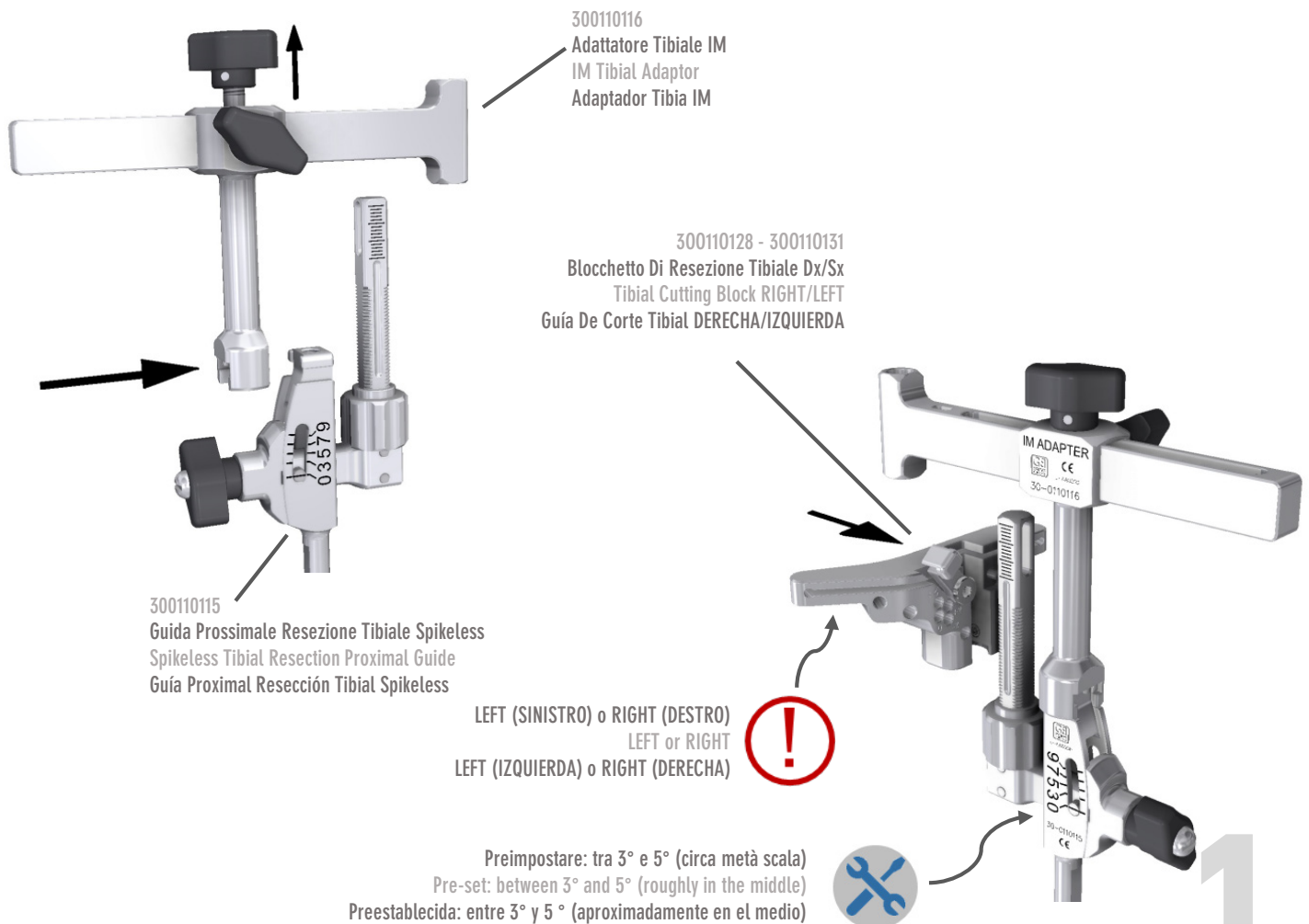
The intra-medullary tibial resection guide consists of three parts: the Spikeless Tibial Resection Proximal Guide (300110115), the appropriate Tibial Cutting Block (RIGHT 300110128, LEFT 300110131) and the IM Tibial Adaptor (300110116).

The guide must be assembled before application. Raise the top knob on the IM Tibial Adaptor to lift the connection pin, gently block the one on the side and mount the Spikeless Tibial Resection Proximal Guide. Release the knob on the side and tighten the top one. Make sure you've chosen the appropriate Tibial Cutting Block (LEFT or RIGHT). Pre-select the desired tibial slope (range: 0°-9°) on the Spikeless Tibial Resection Proximal Guide.

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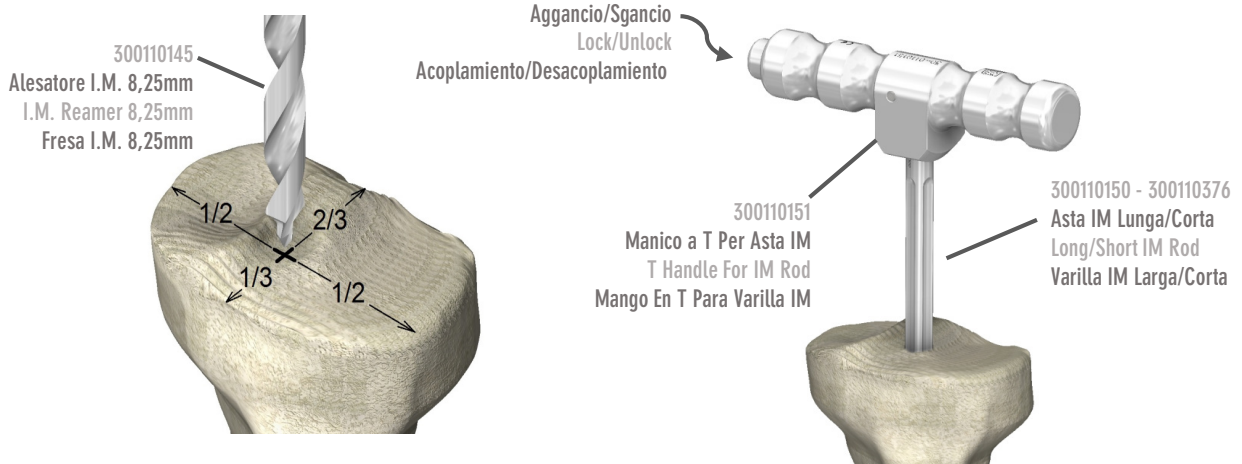
Use the IM Reamer 8,25mm (300110145) to make an intramedullary hole in the tibial channel. The hole must be roughly in the centre of the ML plane and in 1/3 2/3 ratios in AP direction.

Insert the appropriate IM Rod (LONG 300110150, SHORT 300110376) in the hole of the tibial channel with the aid of the T-Handle For IM Rod (300110151). The IM Rod has longitudinal grooves that limit pressurization of the channel and the risk of embolisms.

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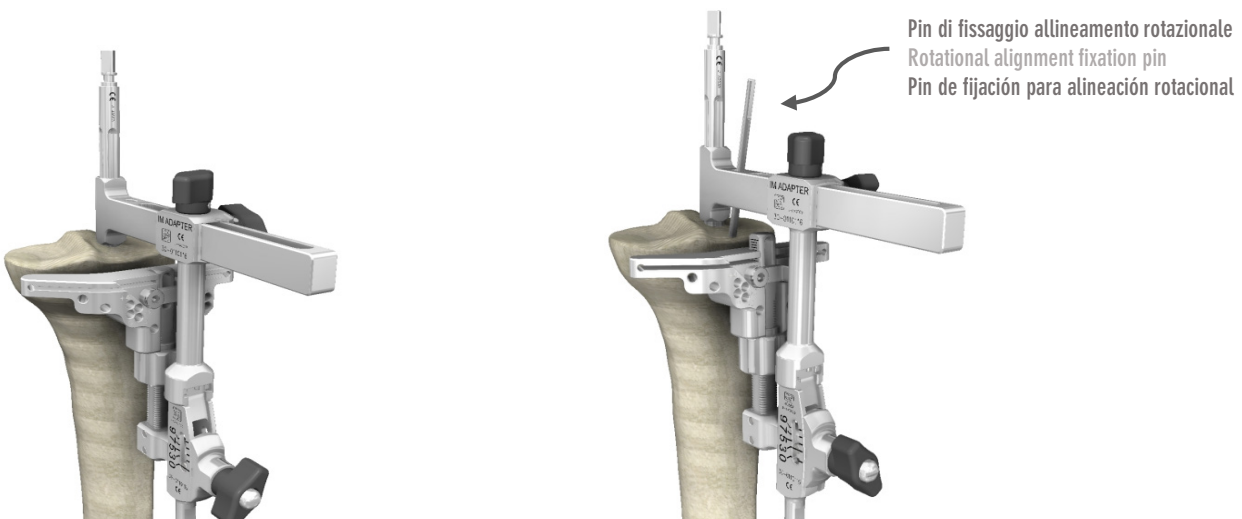
Unlock the T-Handle and mount the intra-medullary tibial resection guide on the IM Rod. Adjust the height of the Tibial Cutting Block to roughly the level of the desired cut. Align the guide on the proximal tibia with the third medial of the tibial tuberosity. Lock it in place using a temporary Pin 3,2mm x 80mm (300110144) in the middle of the IM Adaptor's top slot.

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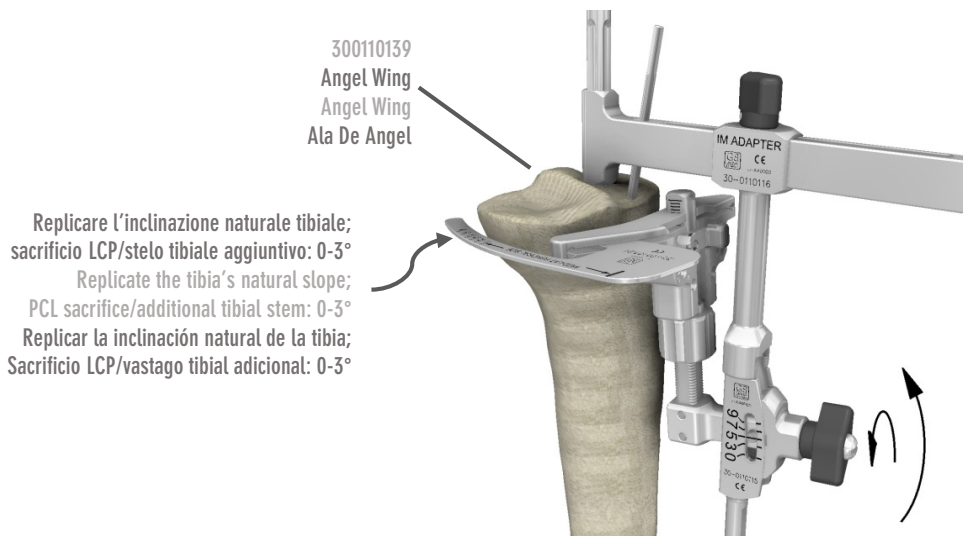
The posterior slope can be adjusted by using the Angel Wing (300110139) and operating the knob on the Spikeless Tibial Resection Proximal Guide. In general, proximal tibial resection should follow the tibia's natural posterior slope.

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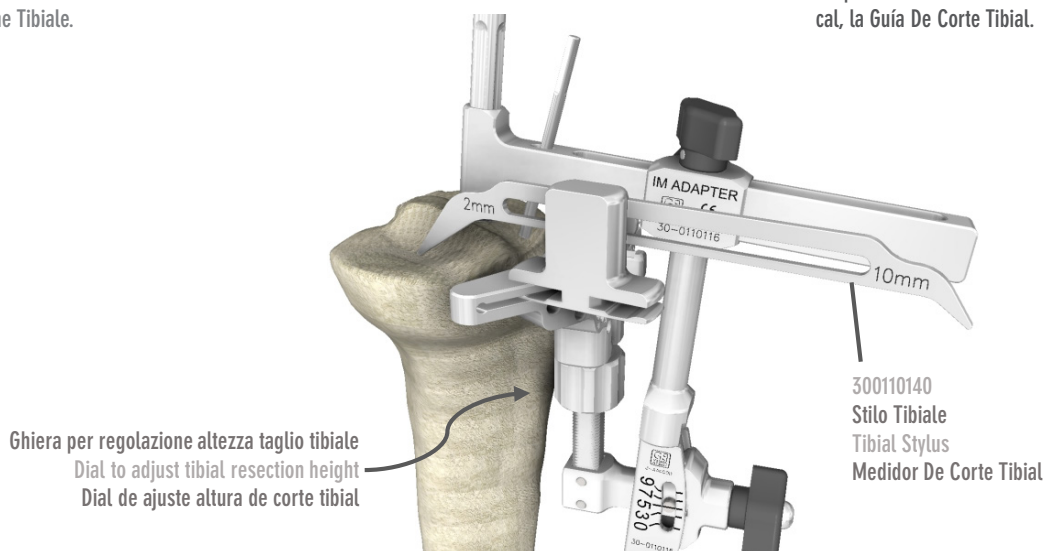
Position the Tibial Stylus (300110140) in the slot of the Cutting Block. The surgeon may choose to use as a cutting reference the lowest point on the most worn side of the tibial plateau and carry out a minimum resection (2mm tip for 2mm resection) or the highest point on the least worn side (10mm tip for 10mm resection). Check that the cut is adequate and if necessary modify the size of the resection by operating the dial and making a micrometric vertical shift of the Tibial Cutting Block.

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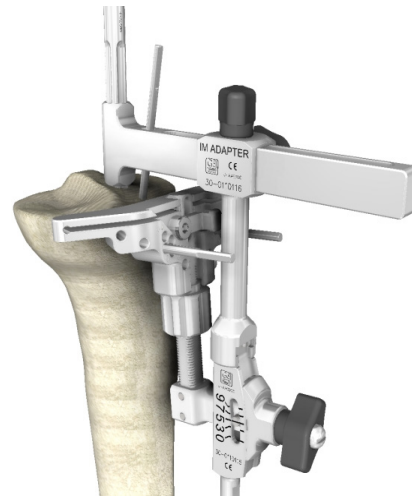
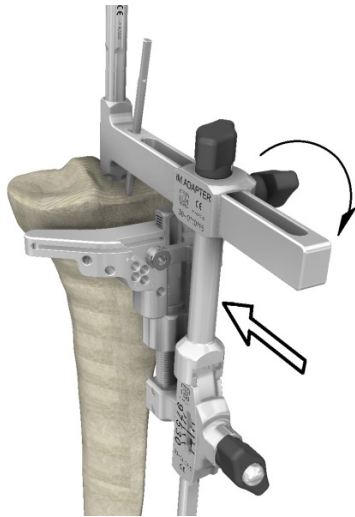
Release the knob on the side of the IM Tibial Adaptor, offer up the Cutting Block to the tibial bone and anchor it with two Pin 3,2mm x 80mm (300110144) in the parallel 0 rows of holes.

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Remove the temporary pin from the slot on the IM Tibial Adaptor.

Remove the guide: raise the Cutting Block trigger and slide the dial downwards. Release the knob on the side of the IM Tibial Adaptor and move it away from the tibia. Extract the assembly by sliding it along the IM Rod.

NB: should the opening of the Cutting Block trigger be obstructed by the top arm of the IM Adaptor, slide the dial downwards and lift the entire assembly till the trigger can be opened. Remove the IM Rod from the tibial channel using the T-Handle.

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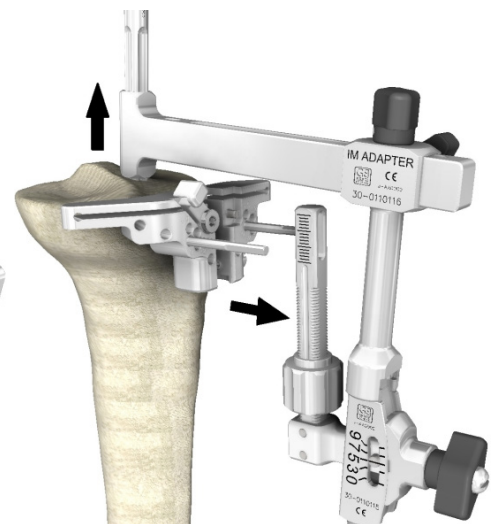
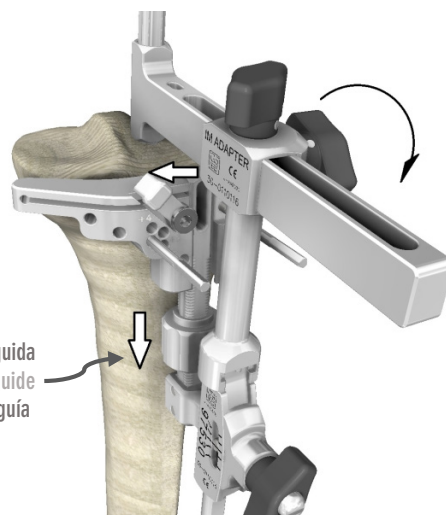
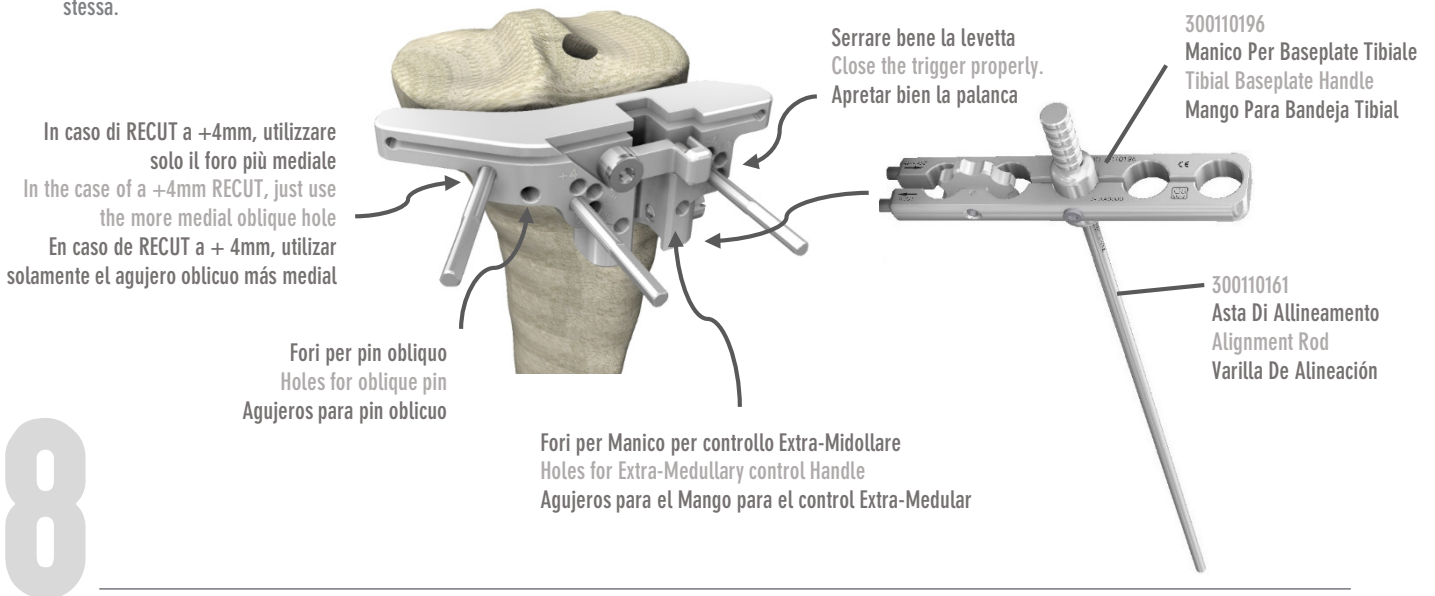
The Cutting Block may be made more stable by inserting a third pin in the oblique holes on the medial side of the Cutting Block, using either the Pin 3,2mm x 80mm (300110144), the Pin with stop 3,2mm x 40mm (300110198) or the Threaded pin Ø3.2mm-Ø4,2mm (300110379). Make sure that the Cutting Block trigger is properly closed.

It's possible to check alignment with the mechanical axis by inserting the Tibial Baseplate Handle (300110196) in the appropriate holes on the Cutting Block and the Alignment Rod (300110161) in the Handle. Make sure the Rod is locked and cannot swing.

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Perform the resection of the proximal tibia. Verify the planarity of the cut by using the Angel Wing and if necessary refinish the cut.

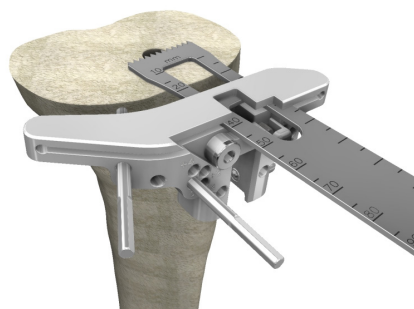
If an oblique pin has been inserted, remove it. Remove the Cutting Block by sliding it outwards. Leave the two parallel pins in for the next steps.

If necessary, a further 2mm or 4mm resection can be made by remounting the Cutting Block on the pins in the +2 or +4 holes.

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After performing the tibial resection, it is possible to evaluate the extrafemoral rotation through appropriate modular spacer blocks. To this end, it is necessary to connect an extrarotation module, from 0 to 6 degrees, to restore the correct tibial thickness: tibial component and polyethylene insert.

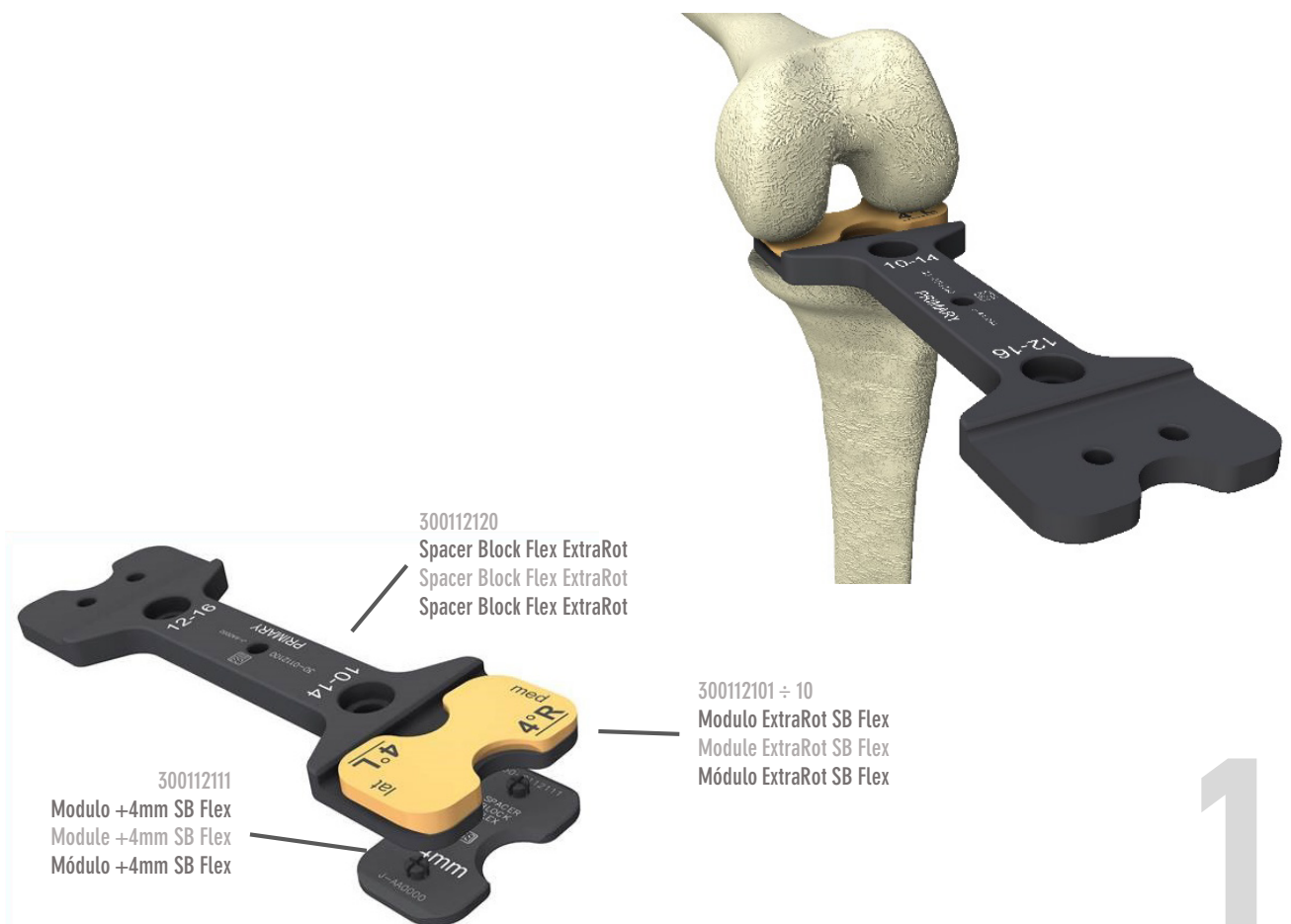
Select a Module ExtraRot SB Flex (300112101÷10), from 0 to 6 degrees, and connect it respecting the side of the leg to be operated, L (Left) or R (Right), to the Spacer Block Flex ExtraRot (300112120). The MED and LAT indications help in the correct positioning. Select the 10 or 12mm side according to the created space; if it is greater, place the Module +4mm SB Flex (300112111) in the lower part to simulate a space of 14 or 16mm.

Position the leg at 90 degrees of flexion. Insert the Spacer Block between the resected tibia and the still intact posterior femoral condyles. Check the ligament balancing and the extrarotation even trying different ExtraRotation Modules.

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In the case of an implant with Mobile UC inserts, the trial reduction can be carried out after tibial preparation.

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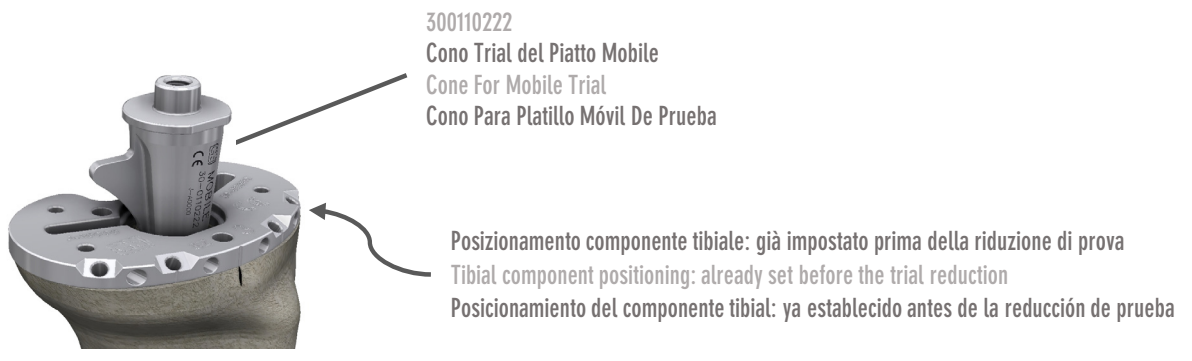
Proceed as described in the TIBIA: PREPARATION chapter, after which it will be possible to insert the Cone For Mobile Trial (300110222) in the conical slot in the Baseplate.

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If Headed Pins were used to fixate the Baseplate, remove them.

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Position the Trial Mobile Insert (300110401 ÷ 36) on the Baseplate by coupling the hole on the bottom surface with the peg of the Cone For Mobile Trial.

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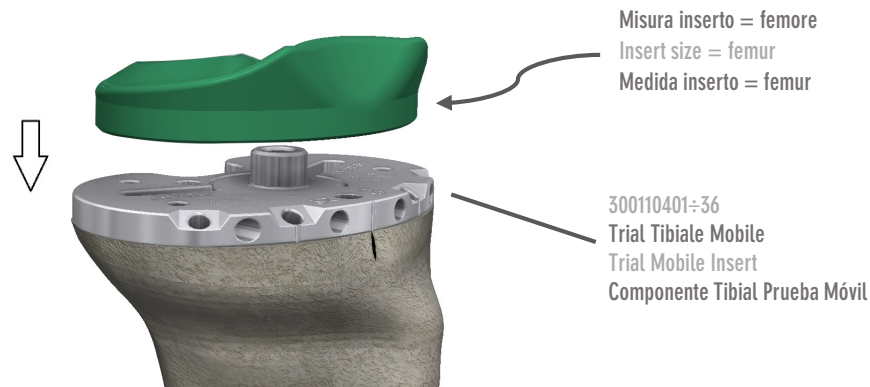
The size of the tibial insert corresponds to that of the femoral component even if Skinny femoral components are subsequently used.

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Carry out the checks described in the TRIAL REDUCTION chapter (section 6).

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By using the trial inserts, assessment of the thickness and the most suitable polyethylene insert may also be carried out after implanting the definitive femoral and tibial components.

The tibial insert must be the same size as the femoral component, even when Skinny femoral components are used. The thickness should correspond to that of the Spacer Block used for extension and flexion gaps.

MOBILE INSERT

Connect the pre-selected Tibial Trial Insert (300110401 ÷ 36; 300111401 ÷ 300111436) and the Conical Adaptor (300110400) and mount them on the Trial Mobile Spacer (sizes 1-2: 300110390; sizes 3-4: 300110391; sizes 5-6: 300110392), which is chosen on the basis of the measurement of the femoral component, and makes it possible to restore the correct thickness of the insert and provide for protection of the articular surface of the definitive tibia. Insert the trial assembly in the definitive tibia.

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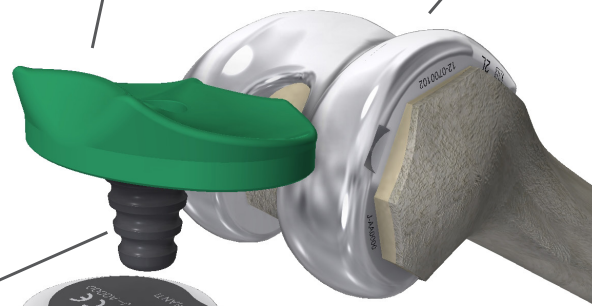
FIXED INSERT

Choose between the options: Posterior Cruciate Ligament retaining CR (300110441 ÷ 76), Ultra Congruent UC (300110481 ÷ 516), Posterior Stabilized (300111301 ÷ 36) or Dynamic Congruence DC (300110521 ÷ 96) for the side of the leg being operated (L: Left; R: Right). Position the trial insert on the definitive tibia and anchor it in its housing with the M5 Screw for Fixed Trial (300110439) and Screwdriver (340085045). The screw is in the femoral instrument kit and coloured purple to distinguish it from the definitive one in the tibial component packets. The M5 Screw for Fixed Trial must not be used for the definitive articular insert.

After checking the stability of the joint in extension and flexion and choosing the most suitable implant, remove the trial insert and proceed to insert the definitive one as described in DEFINITIVE IMPLANTS, sections 3 and 4.

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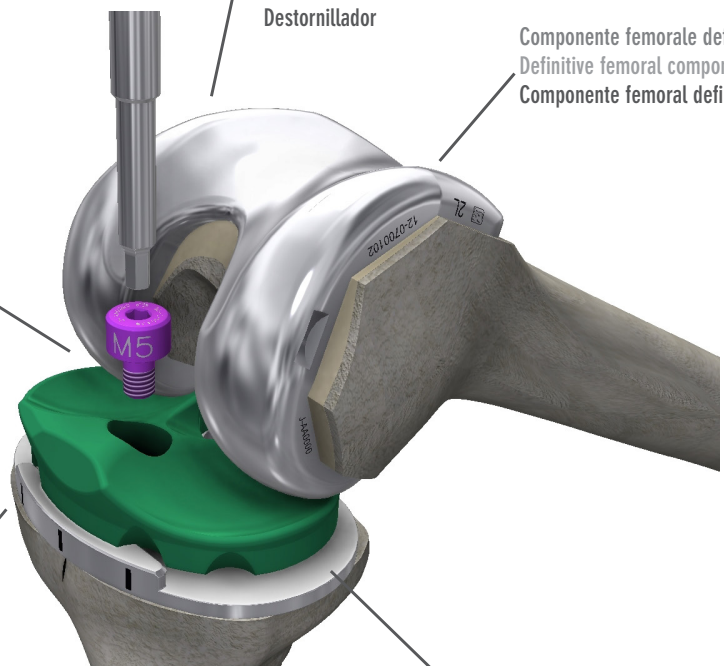
If greater tibial stability is needed, the surgeon may use the additional tibial stems, which come in four lengths (30, 60, 100 and 140 mm) and diameters from 10 to 16 mm and whose dimensions are independent of the size of the tibial component.

When these additional stems are used, and especially the longer ones, it is advisable not to exceed in the posterior slope of the tibial resection, so as to avoid conflict with the anterior diaphyseal bone.

Select the Tibial Stem Reamer of the desired diameter (300110610÷16). Mount the Tibial Stem Reamer Guide (300110600) from the connection side and with the cylindrical body facing the reamer side. Slide it over the Reamer and engage it by the notch for the desired length.

Preparation may proceed in sequence following that of the tibial keel described in the TIBIA: PREPARATION chapter. Here too, if greater stability is needed, the Baseplate can be fixated using the Headed Pins (300110197).

Connect the Tibial Stem Reamer to the drill and move it through the Tibial Keel Guide (300110199) connected to the Baseplate as far as the stop.

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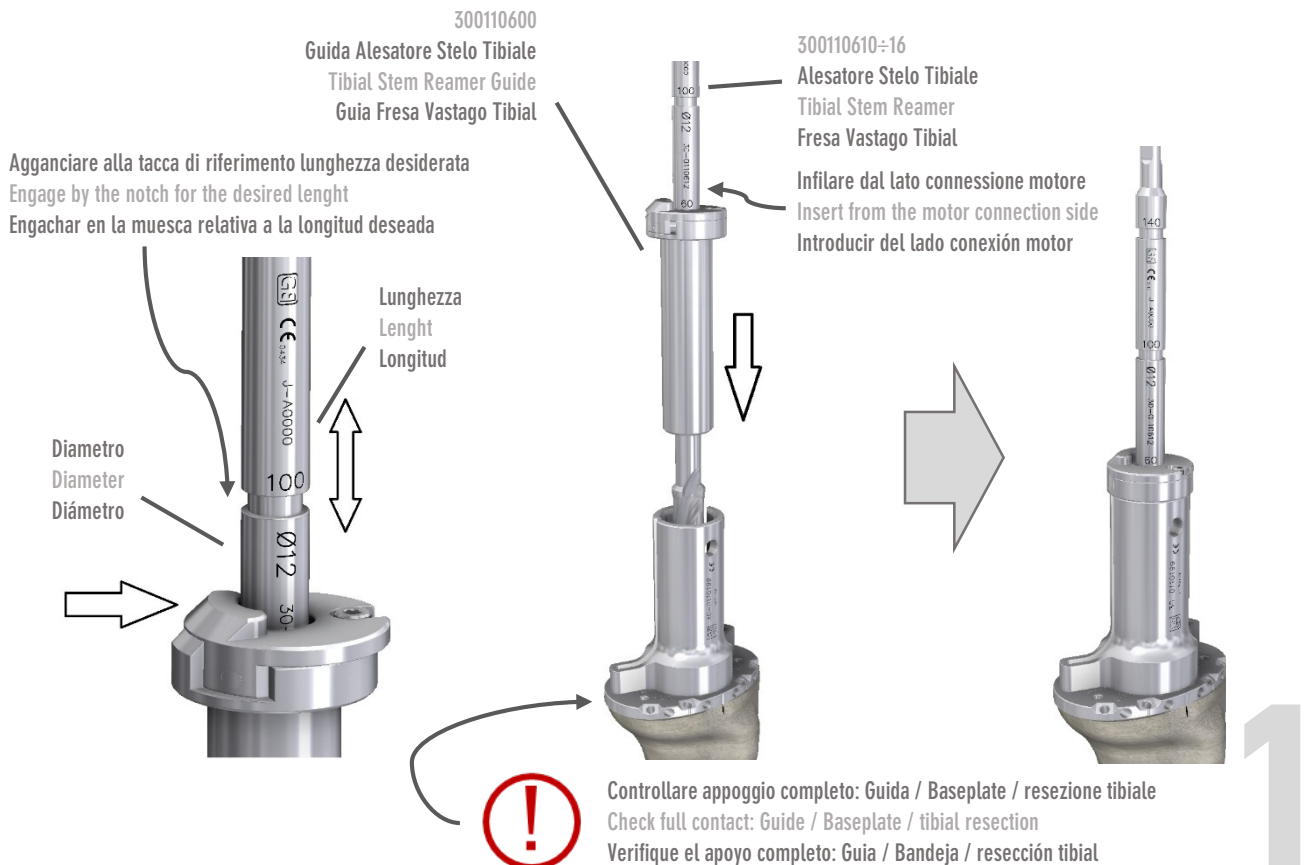
The system does not provide for a reduction with a trial stem, so proceed to remove all the instruments.

NB.: if desired, it is possible to proceed sequentially, both in the lengths and diameters of the Reamers.

Assembling the definitive components:

Open the definitive tibial component, fixed or mobile, and the additional tibial stem, as prepared. Unscrew the tibial stem's plug using the Screwdriver (340085045) and replace it with the additional tibial stem, making sure it's screwed in as far as possible.

If bone cement is being used, it's possible to proceed with cementing up to the level of the connection. The additional stem is meant for use without cement.

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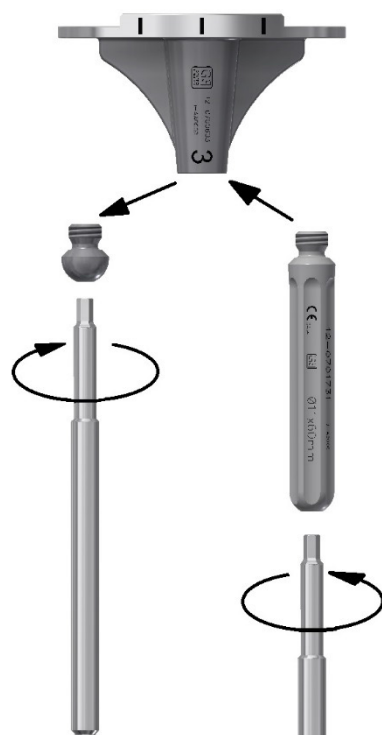
Remove any marginal osteophytes from the patella. If the thickness of the patella is to be measured, use the millimetric scale on the Patella Impaction Clamp (300110210).

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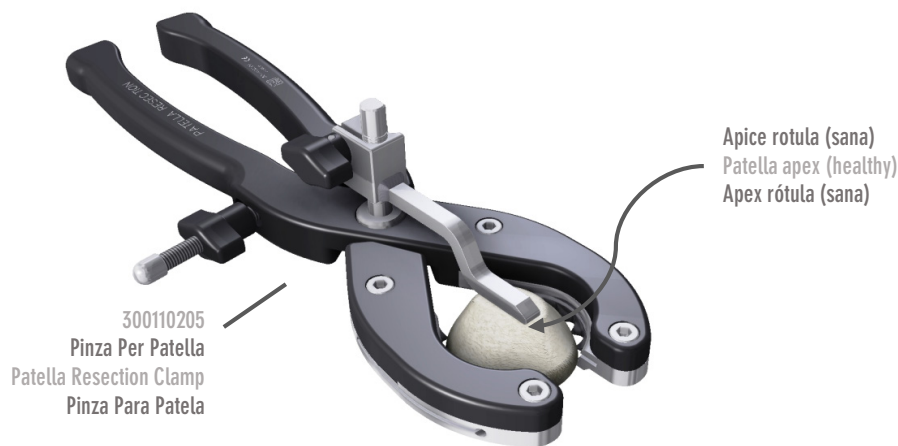
Position the Stylus of the Patella Resection Clamp (300110205) at a resection level of around 8-10mm. Grip the patella in such a way that the stylus is in contact with the apex of the patella.

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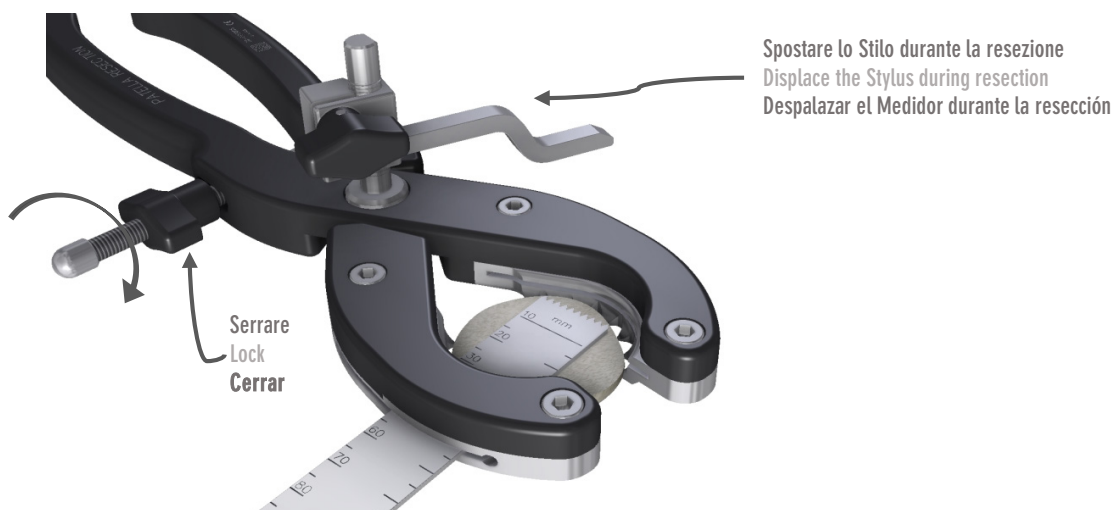
Evaluate the resection using the Angel Wing (300110139) in the lateral and medial slots and if necessary correct it by modifying the resection height of the stylus. Make particularly sure that the thickness of the residual patella is adequate and that the patellar ligament is secured.

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Lock the Patella Resection Clamp, rotate the stylus and perform resection of the patella through the slots.

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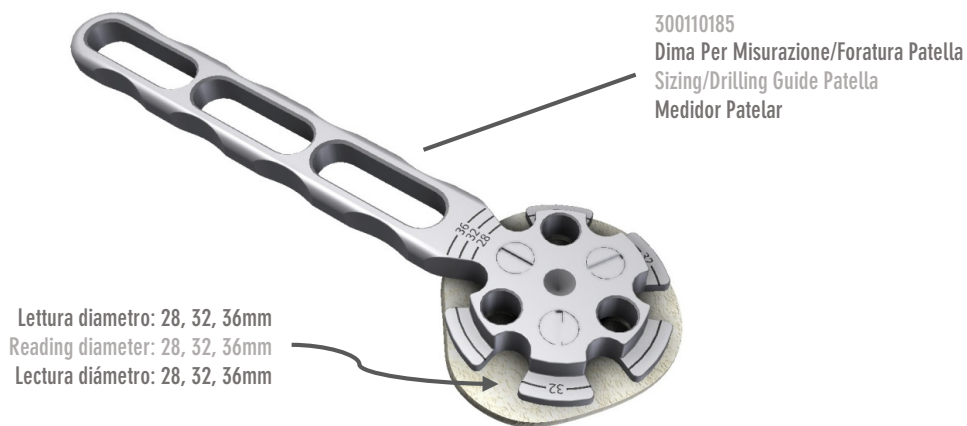
Measure the patella diameter with the Sizing/Drilling Guide Patella (300110185) and choose the appropriate size. The diameter of the patella component is independent of the size of the femoral component.

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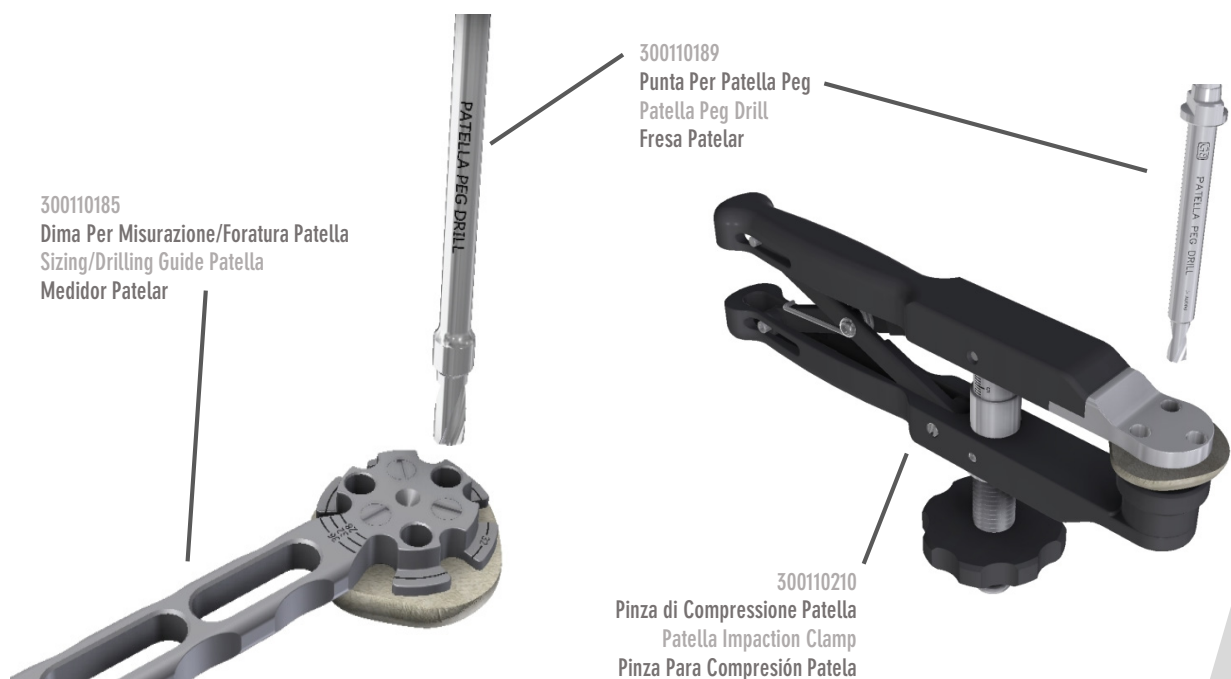
Drill three holes for the pegs of the patellar component using the Patella Peg Drill (300110189), drilling up to the stop. Alternatively, the Patella Impaction Clamp (300110210) may be used.

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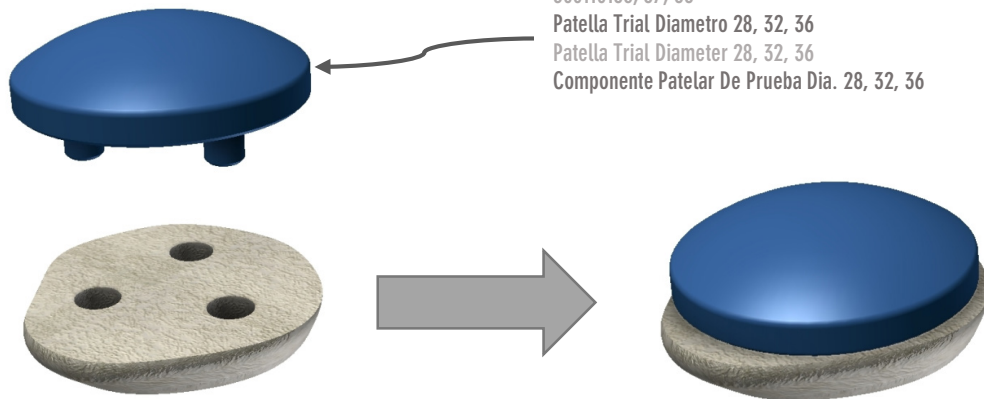
Position the Patella Trial Diameter 28, 32, 36 (300110186, 87, 88) of the chosen size on the patella and press lightly into the three holes. Check the patellar implant in the trial reduction.

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Carefully choose the diameter of the definitive implant and then proceed to apply the cement. Align the three fixation pegs with the three holes drilled in the patella and press lightly to insert the component.

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Complete the positioning with the Patella Impaction Clamp (300110210), whose screw adjustment makes it possible to keep the patellar implant compressed until the bone cement has set definitively.

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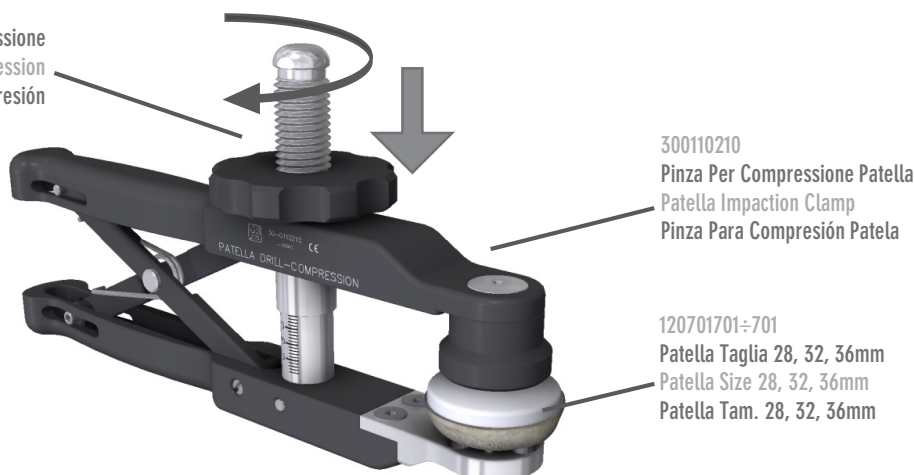
Make sure there is perfect adherence between component and resection and remove any excess cement.

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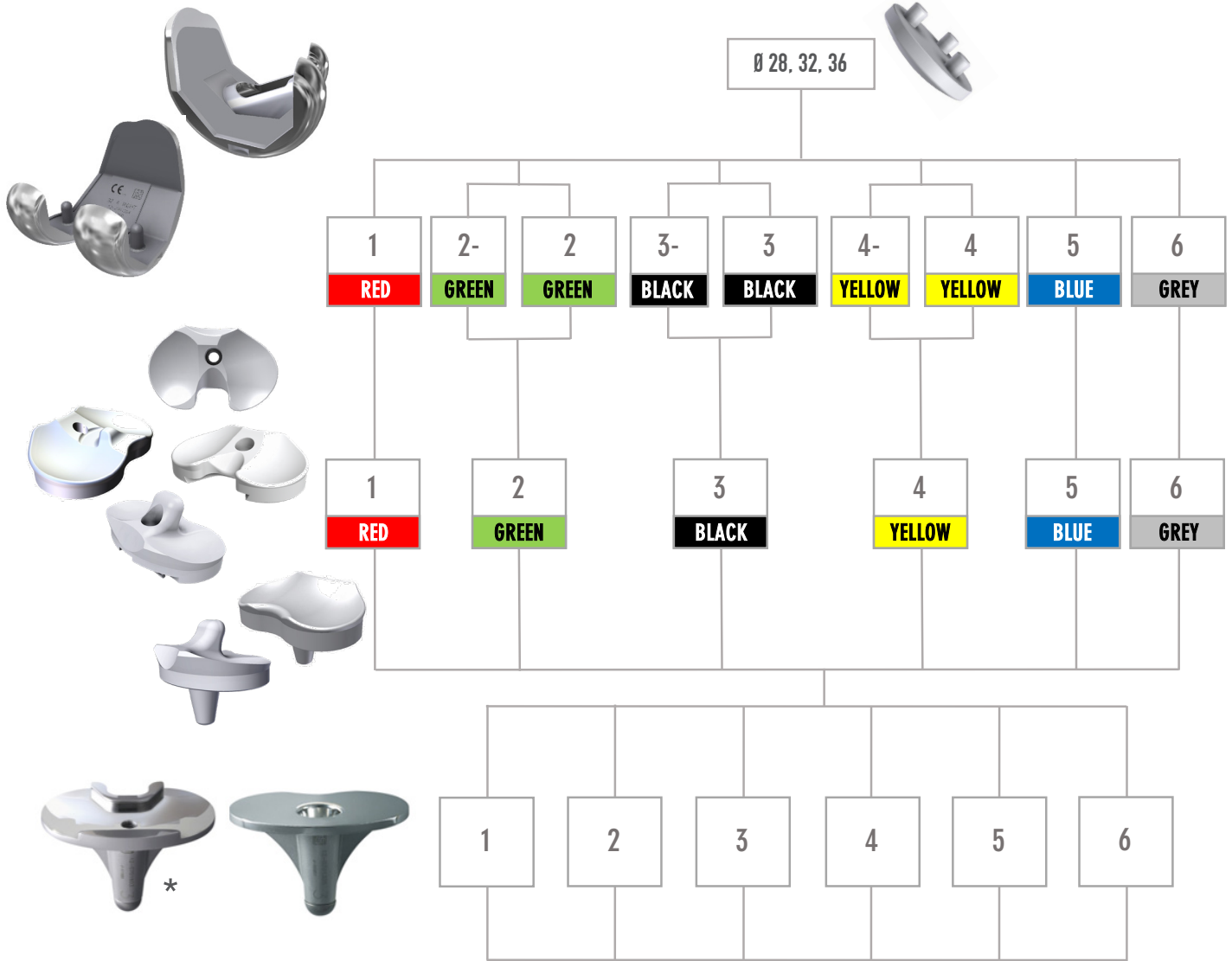
120701701÷701
Patella Taglia 28, 32, 36mm
Patella Size 28, 32, 36mm
Patella Tam. 28, 32, 36mm

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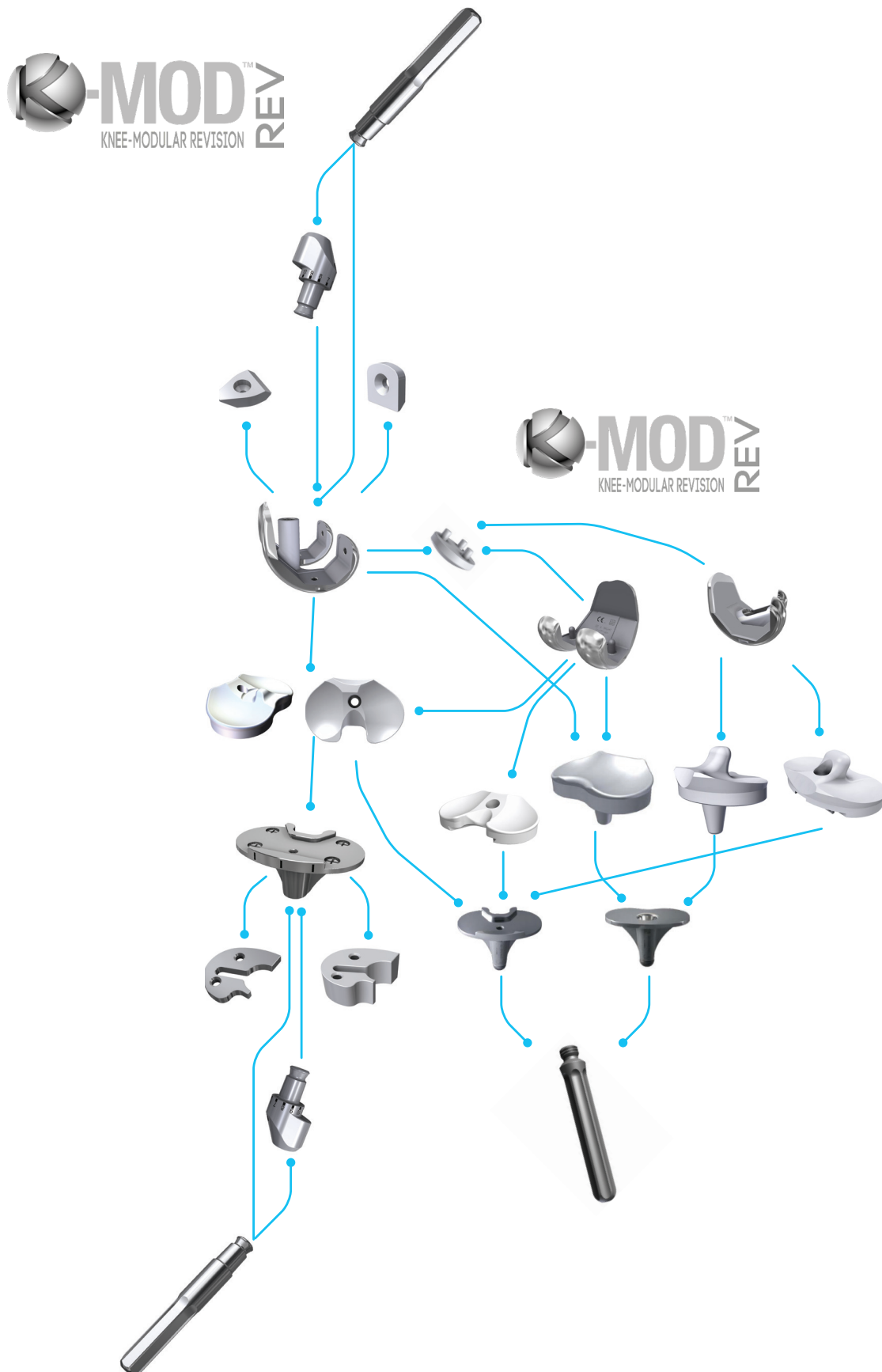
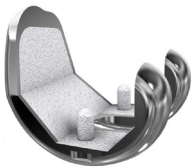
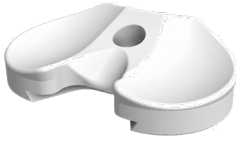
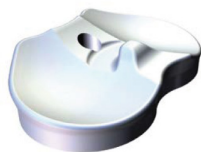
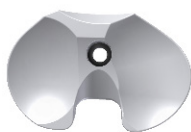
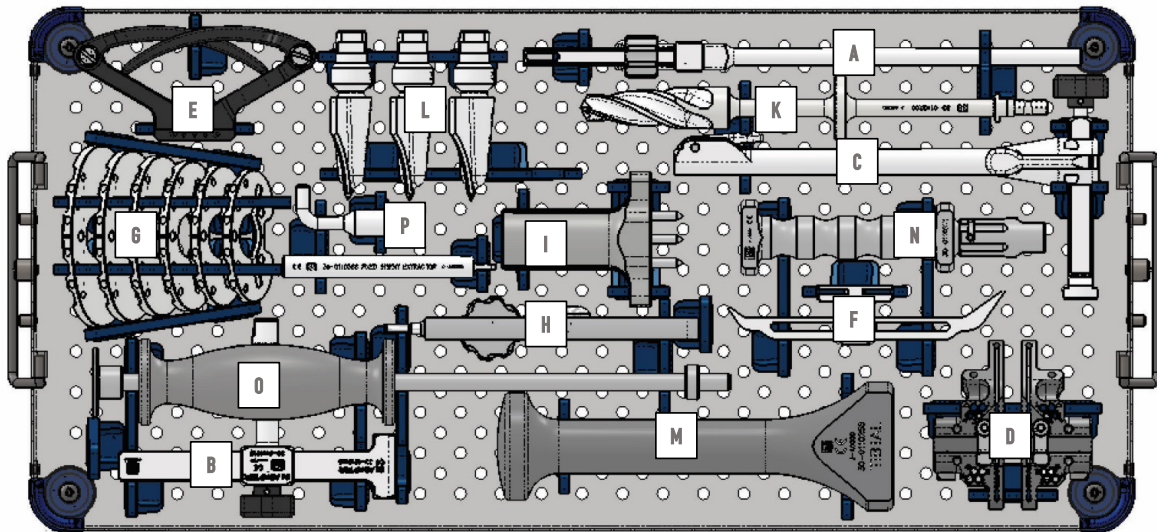
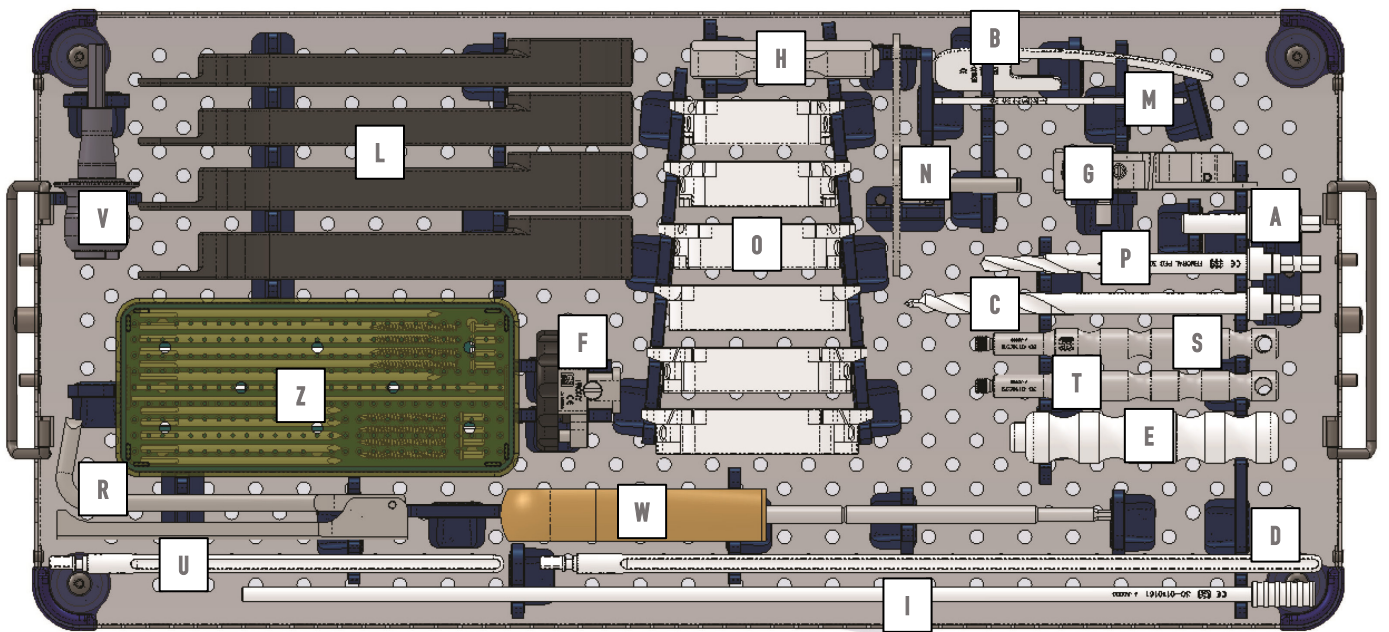
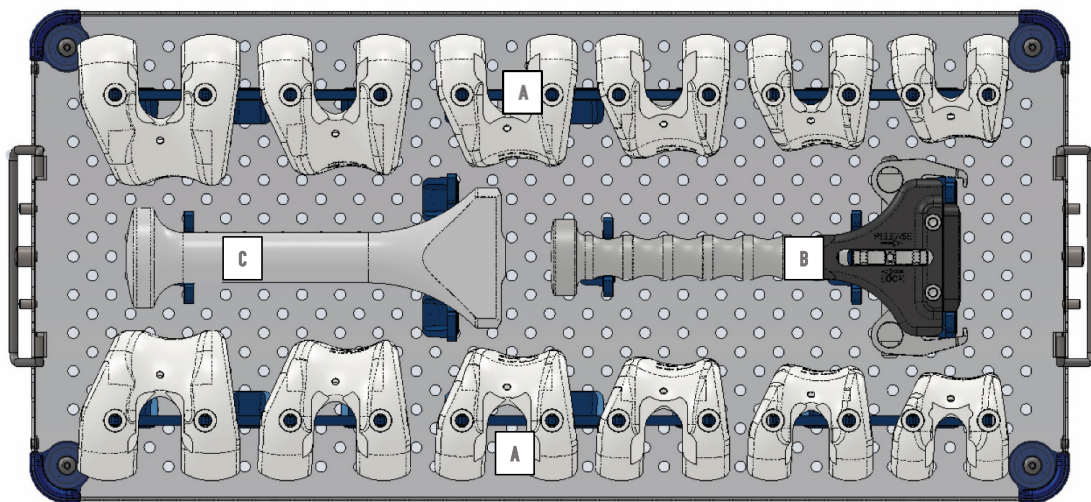
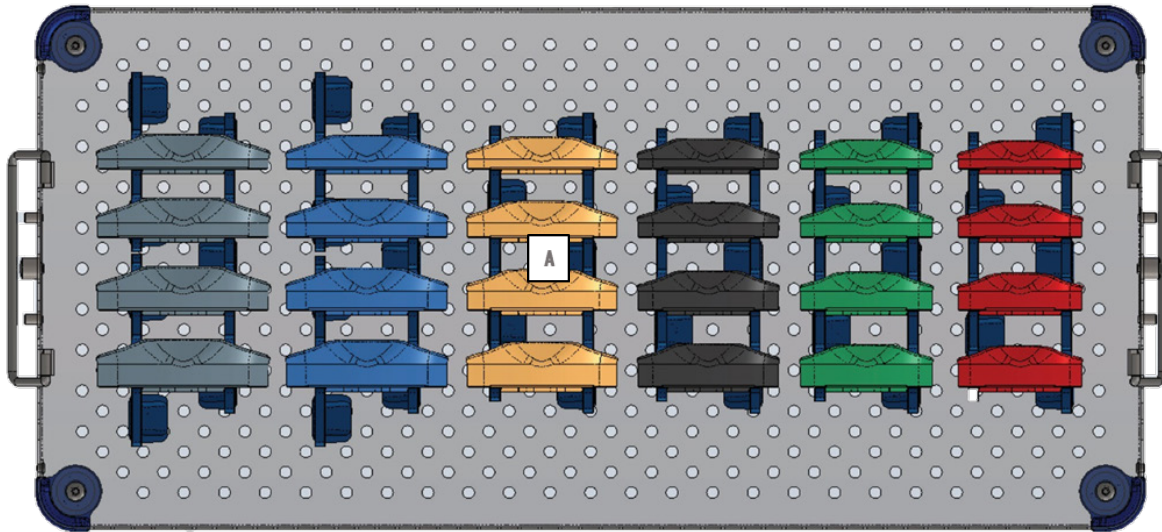
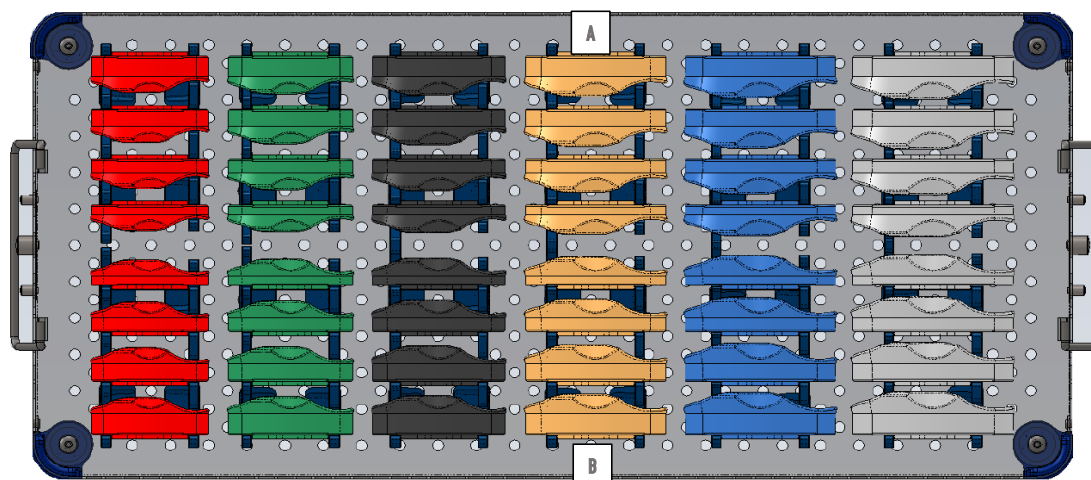
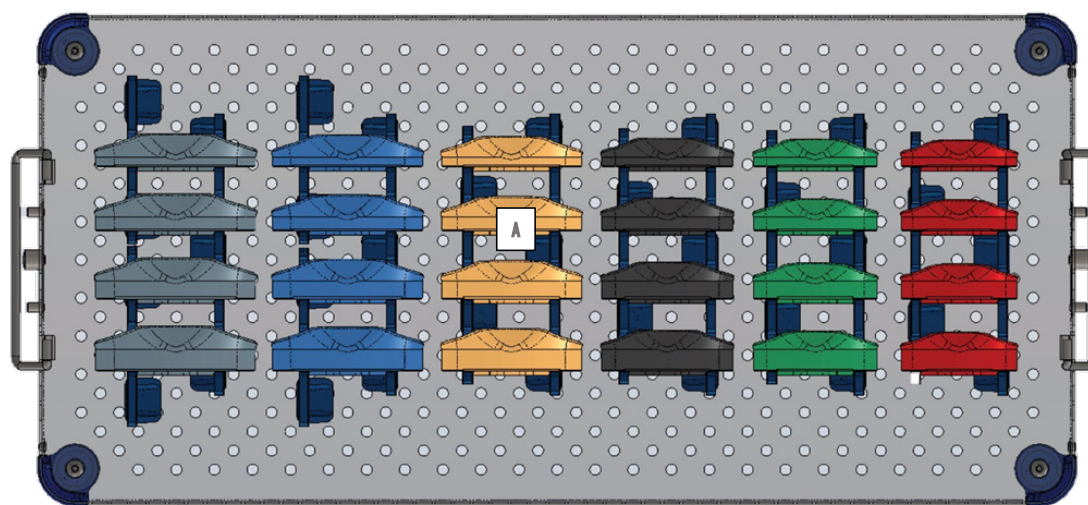
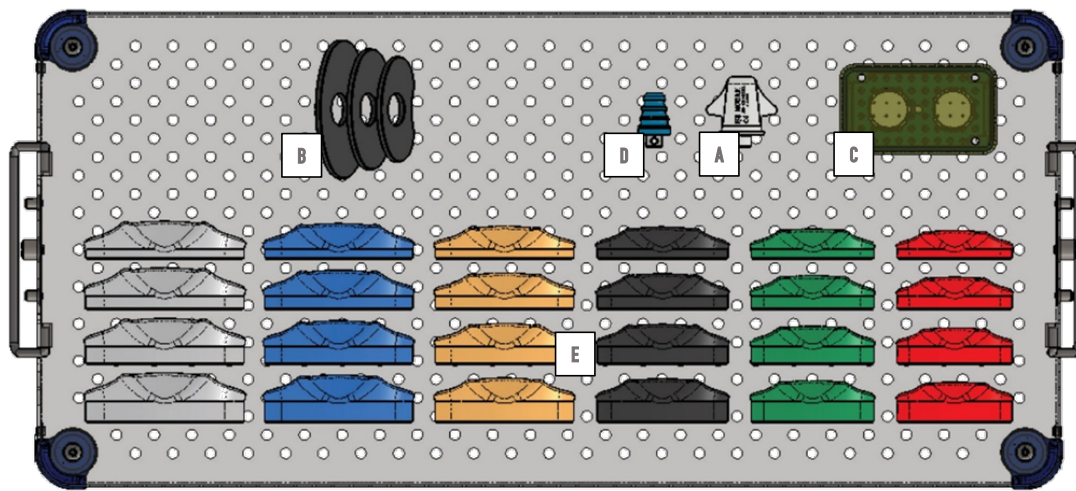
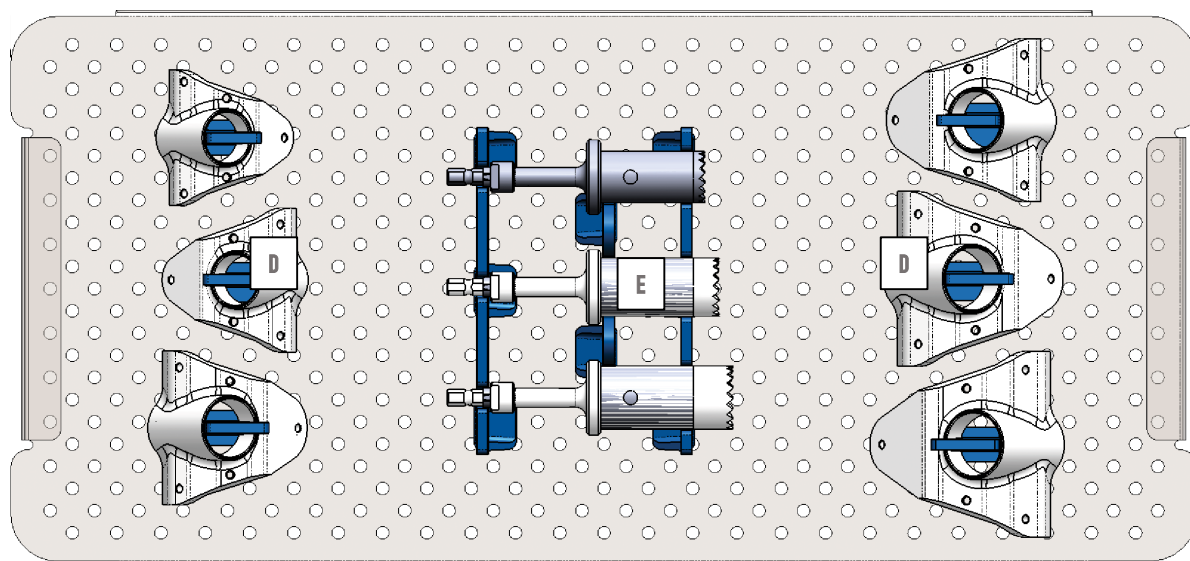
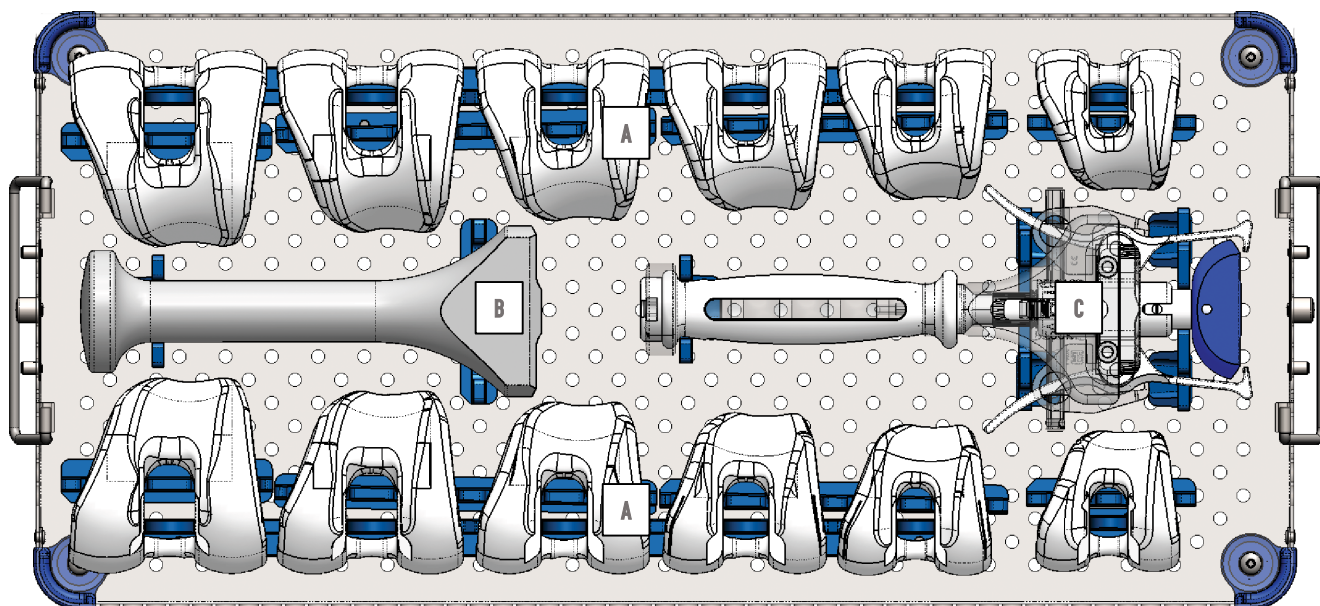
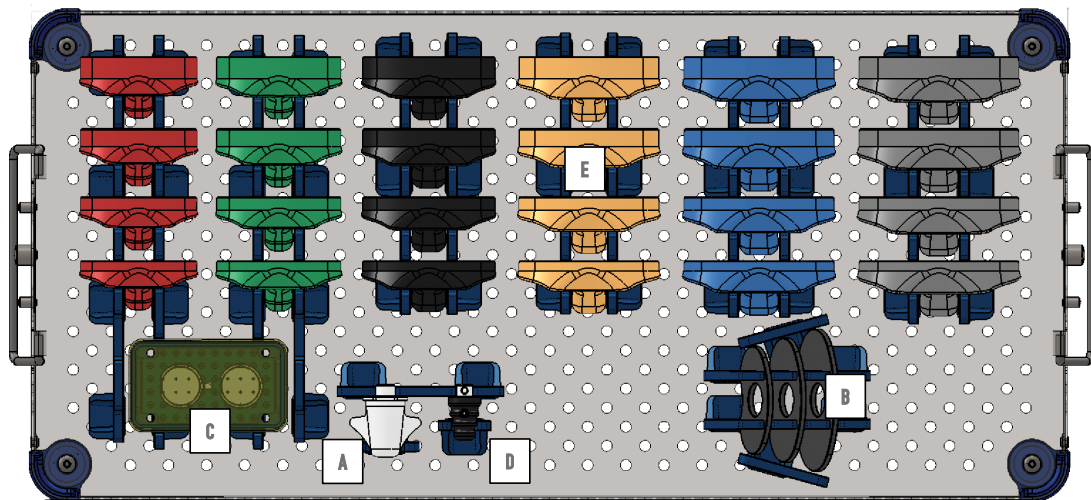
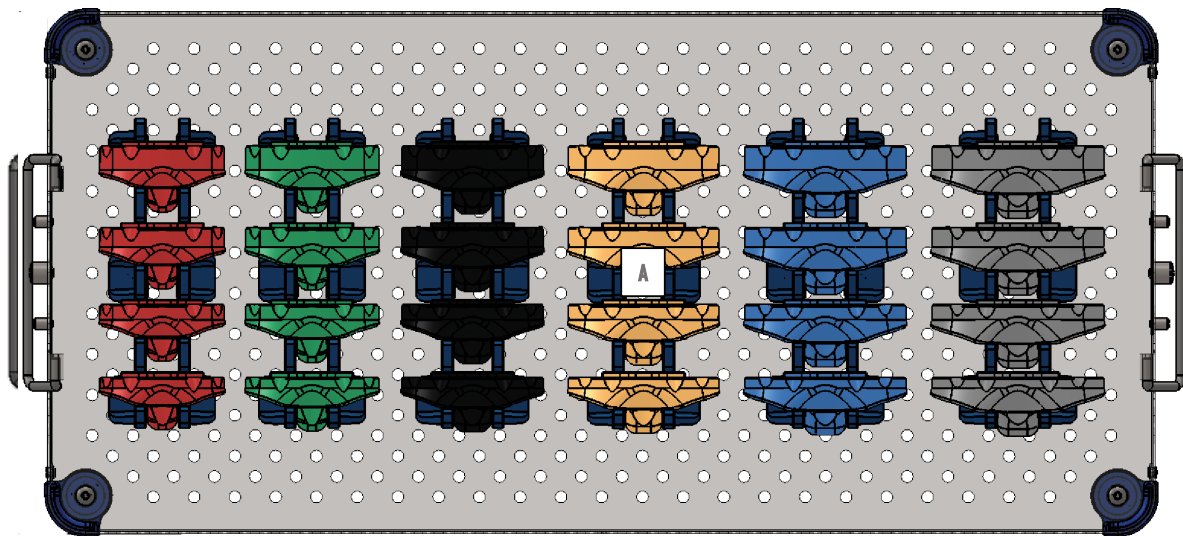
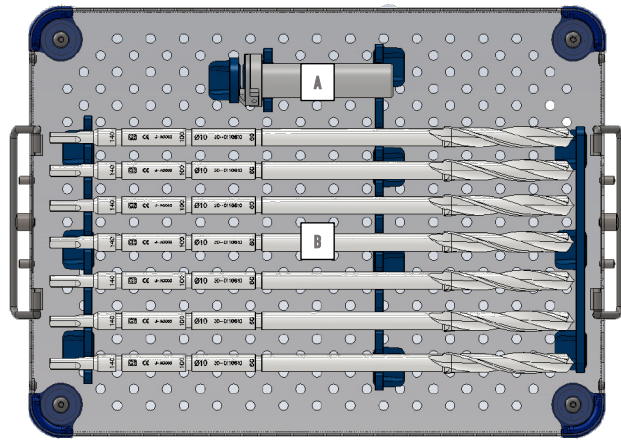
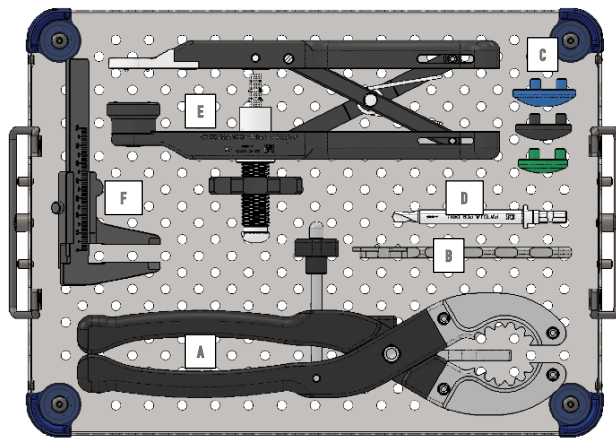
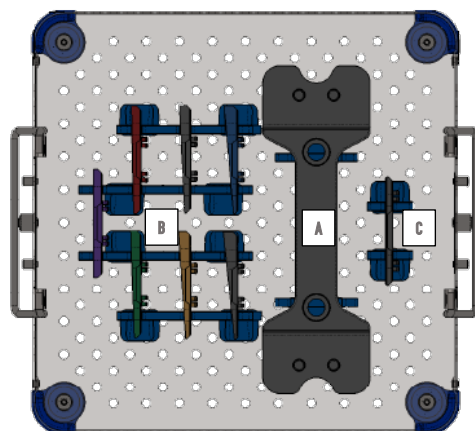
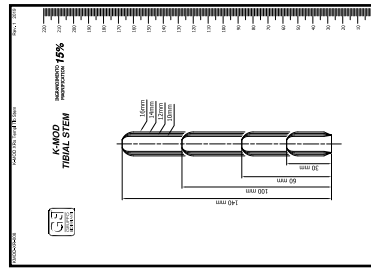
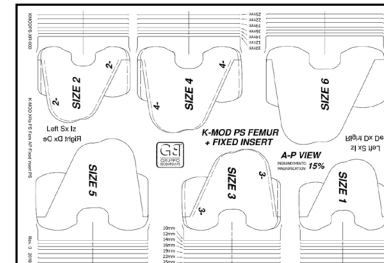
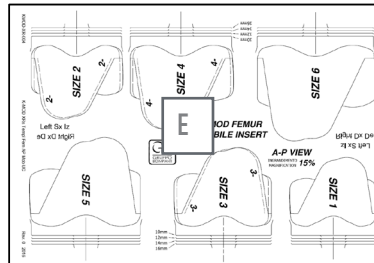
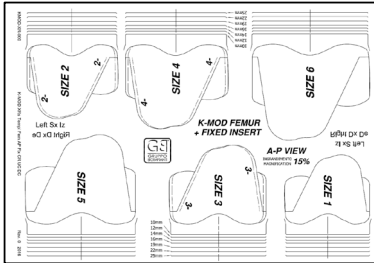
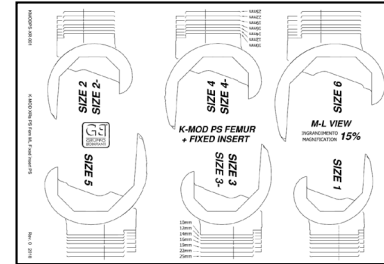
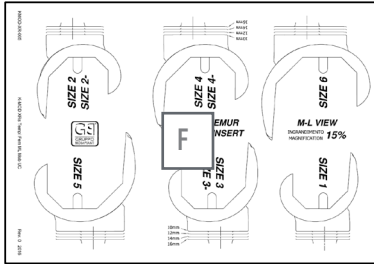
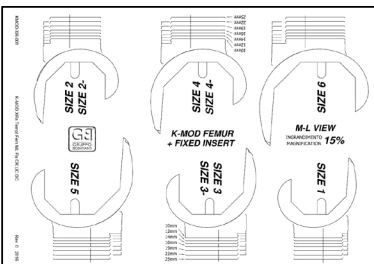
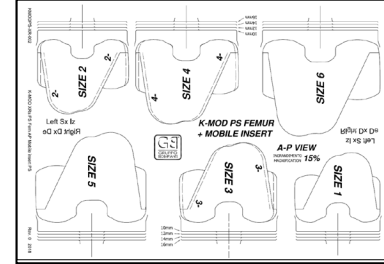
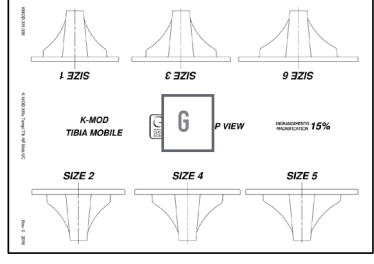
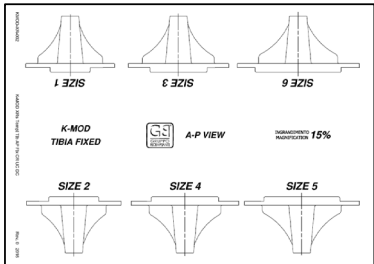
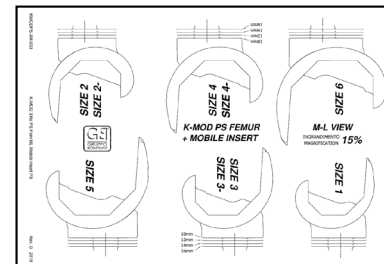
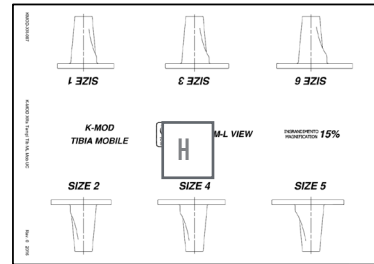
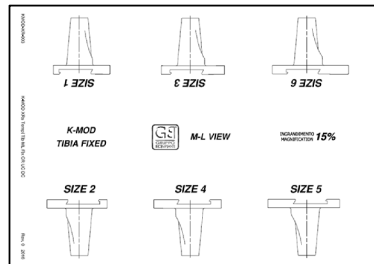
Operating
technique video

EN

Use the QR-Code to display the video that simulates the operating technique

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